# VOCO

## Caries Marker

EN Instructions for use  
MD EU Medical device

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### Product description:

**Caries Marker** contains a colour indicator (acid red) for the staining of infected, carious dentine. In this way, **Caries Marker** allows the visual inspection of carious areas. Healthy, uninjected and remineralisable dentine is not stained, allowing minimally invasive excavation. Uninfected (not stained) dentine should be preserved to the greatest extent possible.

### Indications:

**Caries Marker** dyes carious dental substance

- to reveal remaining carious areas after removal of the visible caries
- to provide a clear boundary between carious and healthy dentine, especially on secondary dentine which is different in shade
- to check if caries has been removed completely
- to trace carious areas which are difficult to see
- to demonstrate carious areas to the patient
- for training and teaching purposes (on extracted teeth)
- for finding obliterated root-canals
- for finding micro-cracks in fillings and margins

### Contraindications:

**Caries Marker** contains propylene glycol. **Caries Marker** should therefore not be used in patients with a known hypersensitivity (allergy) to this ingredient.

### Patient target group:

**Caries Marker** is suitable for use in all patients without any age or gender restrictions.

### Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

### User:

**Caries Marker** should only be used by a professionally trained dental practitioner.

### Use:

Start by removing the softer and significantly discoloured dentine. Coat the cavity with **Caries Marker**, allow to act for 5-10 s (not longer!) and then rinse off with water spray. The liquid can be best applied using **Pele Tim** foam pellets. Carious dentine is stained an intense red. Remove discoloured dentine completely, taking care not to damage the unstained areas of the dentine. Repeat the procedure until no further dentine staining occurs.

**Caries Marker** also stains plaque and residual caries in the tooth enamel red; they should be removed accordingly.

### Warnings, precautionary measures:

- Brief drying of the cavity can increase the intensity of the staining. The staining should not be allowed to act for too long (more than 10 s) and the dye must be completely removed by rinsing the mouth out. Otherwise there is a risk of staining and removing healthy dentine.
- Do not use in the direct vicinity of the pulp so as to avoid the risk of confusing the staining with blood colouration.
- Avoid contact with the skin and clothes (product stains). Use diluted vinegar to clean stained clothes (check for colourfastness first!).
- Avoid undiluted **Caries Marker** coming into contact with mucous membranes.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for the intended purpose.

### Storage instructions and application method:

Store at 4°C-23°C. Bring material to room temperature before use. Do not use after the expiry date.

### Disposal:

Dispose of the product according to local regulations.

### Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Caries Marker** must be reported to VOCO GmbH and the responsible authority.

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VC 60 BB1005 E1 1221 99 © by VOCO