



V-Print c&b temp

Indikationen / Indications

Langzeitprovisorien wie Kronen, Brücken und Mock-ups / Long-term temporaries like crowns, bridges and mock-ups

Druckerhersteller / manufacturer	3D-Druck / 3D printing		Reinigung / Washing		Nachbelichtung / Postcuring		
	Gerät / Device	Schicht / Layer* [µm]	Build Style*	Gerät / Device	Programm / Program	Gerät / Device	Programm / Program
VOCO ¹	SolFlex 170 [SMP] SolFlex 350 [SMP] SolFlex 650 [SMP] SolFlex 170 HD	100	Gentle	–	Pinselreinigung / cleaning with brush	OtoFlash G171 (280N2 Schale / tray)	2 × 2000 flashes
W2P ¹	SolFlex 170 [Plus] SolFlex 350 [Plus] SolFlex 650 [Plus] SolFlex 163 FullHD SolFlex 250 UHD	100	Gentle	–	Pinselreinigung / cleaning with brush	OtoFlash G171 (280N2 Schale / tray)	2 × 2000 flashes
ASIGA	MAX UV PRO 4K	100	x	–	Pinselreinigung / cleaning with brush	OtoFlash G171 (280N2 Schale / tray)	2 × 2000 flashes
Rapid Shape	D20 II D30 II D40 II D10+ D20+ D30+ D50+	50	x	–	Pinselreinigung / cleaning with brush	OtoFlash G171 (280N2 Schale / tray)	2 × 2000 flashes
DMG	3Delite 3Demax	50	x	–	Pinselreinigung / cleaning with brush	OtoFlash G171 (280N2 Schale / tray)	2 × 2000 flashes
Shera	SHERAprint 20 SHERAprint 30 SHERAprint 40 SHERAprint 20+ SHERAprint 30+ SHERAprint 50+	50	x	–	Pinselreinigung / cleaning with brush	OtoFlash G171 (280N2 Schale / tray)	2 × 2000 flashes
Straumann	P20 II P30 II P40 II P10+ P20+ P30+ P50+	50	x	–	Pinselreinigung / cleaning with brush	OtoFlash G171 (280N2 Schale / tray)	2 × 2000 flashes
Dentalwings	D20 II D30 II D40 II D10+ D20+ D30+ D50+	50	x	–	Pinselreinigung / cleaning with brush	OtoFlash G171 (280N2 Schale / tray)	2 × 2000 flashes
Microloy	Versus	100	x	–	Pinselreinigung / cleaning with brush	OtoFlash G171 (280N2 Schale / tray)	2 × 2000 flashes
MiiCraft	Prime 150Y Ultra 125Y / 150Y Hyper 125Y Profession 250Y	100	x	–	Pinselreinigung / cleaning with brush	OtoFlash G171 (280N2 Schale / tray)	2 × 2000 flashes
Ackuretta	SOL	Fine	x	–	Pinselreinigung / cleaning with brush	OtoFlash G171 (280N2 Schale / tray)	2 × 2000 flashes
Ackuretta	SOL Plus	Standard	x	–	Pinselreinigung / cleaning with brush	OtoFlash G171 (280N2 Schale / tray)	2 × 2000 flashes
Shining 3D	AccuFab-L4D	100	x	–	Pinselreinigung / cleaning with brush	OtoFlash G171 (280N2 Schale / tray)	2 × 2000 flashes
UNIZ Technology	NBEE	100	x	–	Pinselreinigung / cleaning with brush	OtoFlash G171 (280N2 Schale / tray)	2 × 2000 flashes
UnionTech	E128 E230	100	x	–	Pinselreinigung / cleaning with brush	OtoFlash G171 (280N2 Schale / tray)	2 × 2000 flashes
DENTAL MAKERS	LightBuilder 4K	100	x	–	Pinselreinigung / cleaning with brush	OtoFlash G171 (280N2 Schale / tray)	2 × 2000 flashes

x = Nicht anwendbar / not applicable *empfohlene/r Schichtstärke/Build Style / recommended layer thickness/build style

¹ nur mit PowerVat / only with PowerVat

VOCO GmbH
Anton-Flettner-Str. 1-3
27472 Cuxhaven
Germany

Phone +49 (4721) 719-0
Fax +49 (4721) 719-140
e-mail: marketing@voco.com
www.voco.dental



THE DENTALISTS