VOCO
CleanJoy®

EN Instructions for use
MD EU Medical device

Product description:

CleanJoy is a tooth cleaning and polishing paste used for professional tooth cleaning and polishing within the framework of a prophylactic treatment. The fluoride-containing paste (700 ppm fluoride) is offered in three different cleaning strengths, so that, depending on the degree of contamination, each patient can receive a treatment which addresses their individual situation. The safe application and easy identification of a particular paste with its own distinct level of cleaning is ensured by the colour coding of the tubes and pastes:

Red (RDA* = 195) = High cleaning strength
Yellow (RDA* = 127) = Medium cleaning strength
Green (RDA* = 16) = Polishing paste
*RDA value as per DIN EN ISO 11609

Indications:

- Removal of surface, soft and hard plaque.
- Removal of extrinsic discolouration (e.g., staining from coffee, tea or tobacco).
- Polishing tooth surfaces within the framework of a professional tooth cleaning, prior to bleaching and following the removal of orthodontic devices.
- Cleaning and polishing restorations within the framework of a professional tooth cleaning.
- Removal of residues of temporary luting material prior to definitive bonding (cementation).

Contraindications:

CleanJoy contains artificial flavouring. **CleanJoy** should not be used in patients with a known hypersensitivity (allergy) to these constituents.

Patient target group:

CleanJoy is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

CleanJoy should only be used by a professionally trained dental practitioner.

Use:

Processing:

- Remove the tube cap and dispense some of the material onto a mixing pad. **CleanJoy** can be applied with any commercially available cup or small brush. After use, replace the tube cap.
- In order to avoid potential heat generation and splashing, an appropriate rotational speed should be used when working with the paste (2,000 to 3,000 rpm).
- A final treatment with the polishing paste (green paste) must follow each and every application of a higher strength cleaning paste (red or yellow paste). The polishing paste can also be used following treatment with a powder jet device.
- After the application, rinse the cleaned surfaces thoroughly with water, remove remaining material by means of suction and ask the patient to rinse out their mouth thoroughly.

Warnings, precautionary measures:

- In case of contact with the eyes, rinse thoroughly with water and consult a doctor.
- An excessive intake of fluoride may cause nausea. Do not swallow **CleanJoy**!
- The information and/or advice we provide does not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Storage:

Store at 39°F to 73°F (4°C to 23°C). Do not use **CleanJoy** after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **CleanJoy** must be reported to VOCO GmbH and the responsible authority.

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VC 60 AA2090 E1 1123 99 © by VOCO