VOCO Twinky Star®

EN Instructions for use
MD EU Medical device

In accordance with DIN EN ISO 4049

Product description:

Twinky Star is a coloured compomer with glitter effect. The radiopaque filling material is suitable for cavities in deciduous teeth. It contains 78 % w/w inorganic fillers and is cured with halogen light /LED light (blue light).

Twinky Star is available in caps for direct application and is used in combination with a dentin/enamel bond.

Shades:

Gold, Silver, Blue, Pink, Green, Orange, Lemon, Berry

Indications:

– Fillings in deciduous teeth

Contraindications:

Twinky Star contains methacrylate and BHT. **Twinky Star** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

Patient target group:

Twinky Star is suitable for use in all patients with first dentition of any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Twinky Star should only be used by a professionally trained dental practitioner.

Use:

Preparation:

Clean the teeth being treated. If applicable, mark occlusal contact points. Bring material to room temperature before use.

Shade selection:

Choose one of the 8 colours indicated in the shade guide.

Cavity preparation:

In principle, the cavities should be prepared according to the rules for adhesive restorative treatment; preparation should be minimally invasive in order to protect healthy dental hard tissue.

Follow special preparation forms for deciduous teeth!

Cleaning:

- Remove any residues in the cavity with a water jet.
- Avoid contamination with blood or saliva after cleaning.
- Remove any excess water with a faint airjet. Do not overdry the dentine! The dentine surface should be moist but not too wet (wet bonding).
- Secure a dry working field.

Pulp protection:

It is recommended to apply a calcium hydroxide lining in areas close to the pulp.

Bonding materials:

Twinky Star is applied with a dentine enamel bond, using the adhesive technique. Any light-curing bonding materials can be used. Observe the relevant instructions for use during preparation (etching technique) and processing.

Application:

Insert Caps into the opening of the dispenser (follow instructions for use). Turn the cap into desired direction and remove the protective top. Apply **Twinky Star** directly into the cavity by slowly and evenly pressing the levers of the dispenser together.

Fillings over 2 mm in depth should be applied and cured in layers.

Use transparent strips and crowns or light-wedges. Ensure a tight marginal seal.

Light-curing:

Conventional polymerisation devices are suitable for light-curing the material. Depending on light output, polymerisation times are as follows:

LED-/halogen lamp	all colours
≥ 1000 mW/cm ²	20 s
500 - 1000 mW/cm ²	40 s

Bring the light aperture as close as possible to the light-curing unit as otherwise the curing will be less effective. Inadequate curing can cause discolouration and complaints.

Finishing:

After removing the matrices, the filling can be trimmed and polished immediately while being cooled (e.g. fine or extra-fine diamond burs, polisher). The tooth should be fluoridated as a final step.

Warnings, precautionary measures:

- Phenolic substances, especially preparations containing eugenol or thymol, inhibit the curing of **Twinky Star**. Therefore, do not use zinc oxide eugenol cements or other eugenol-containing substances in combination with **Twinky Star**.
- Each **Twinky Star** Caps unit should be used for one patient only and should be applied exerting uniform but not excessive pressure.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Constituents (in descending order according to content):

Barium aluminium borosilikatglas, strontium aluminium borosilikatglas, BisGMA, UDMA, fumed silica, TEGDMA, fluorosilikatglas, initiators, stabilisers, pigments, glimmer

Storage:

Storage at 4°C-23°C. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Twinky Star** must be reported to VOCO GmbH and the responsible authority.

Note:

The Summary of Safety and Clinical Performance of **Twinky Star** can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>). Detailed information can also be found at www.voco.dental.

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VC 60 BB1680 E1 0823 99 © by VOCO