



Easy  
application  
into the  
sulcus

The cord from the capsule!

## VOCO Retraction Paste

ASTRINGENT PASTE FOR AN EFFECTIVE TEMPORARY  
WIDENING AND DRY GINGIVAL SULCUS

## The cord from the capsule

### ASTRINGENT PASTE FOR AN EFFECTIVE TEMPORARY WIDENING AND DRY GINGIVAL SULCUS

Perfect impressions, moisture control and gentle soft tissue management are essential for exact crown margins, perfect marginal adaption, long-lasting and aesthetic restorations.

#### Indispensable in the practice!

The VOCO Retraction Paste is your indispensable helper for the effective temporary widening and dry gingival sulcus. This creates the condition for successful conventional or digital impressions.

VOCO Retraction Paste can be easily and hygienically applied into the sulcus directly from the *SingleDose* Cap, an innovation from the VOCO engineers, and is sufficient for up to 3 sulci. Thanks to a soft and flexible plastic cap, the application occurs more tissue-conserving and thus more comfortable for the patients in comparison to an application with metal cannulas or placing retraction cords. The viscosity of the paste has been developed according to the clinical treatment requirement. Depending on how much pressure is applied to the paste, it has different viscosity levels, the so-called Viscosity Change. The paste can be easily expressed, flows properly and can be applied precisely with low pressure. The paste is stable in the sulcus – this leads to an effective temporary widening.

The astringent reacting, aluminium chloride containing paste stops any bleeding and displaces moisture. After a short application time of 1 - 2 minutes, the high-contrast VOCO Retraction Paste can be sprayed off easily. The paste can be used as an alternative or in combination with e.g. retraction cords. It is up to you whether you take a classic impression with impression material or a digital impression with an intraoral scanner.

#### Indications

For temporary retraction of the marginal gingiva to provide a dry gingival sulcus when the periodontium is healthy prior to, e.g.

- analogue or digital impressions
- cementation of temporary and permanent restorations
- preparation of class II and V fillings

#### Advantages

- **Thin cannula with flexible tip** – easy and pinpoint application into the sulcus
- **Viscosity Change** – paste consistency varies during application and sulcus widening
- **Good visibility** – contrasty to the gingiva
- **A clean product** – quick and easy to spray off

#### Presentation

- REF 1012 Caps 25 x 0.3 g
- REF 1013 Caps 100 x 0.3 g



VOCO GmbH  
Anton-Flettner-Straße 1-3  
27472 Cuxhaven  
Germany  
  
Tel.: +49 (0) 4721-719-0  
Fax: +49 (0) 4721-719-140  
  
info@voco.com  
www.voco.dental

Available from: