# Caps Warmer

USER MANUAL  
BENUTZERHANDBUCH  
MANUEL D'UTILISATION  
MANUAL DEL USUARIO  
MANUALE D'USO

## CONTENT

USER MANUAL.....	4
BENUTZERHANDBUCH .....	8
MANUEL D'UTILISATION.....	12
MANUAL DEL USUARIO.....	16
MANUALE D'USO .....	20

## IMPORTANT INFORMATION BEFORE FIRST USE

### 1. Foreword

Thank you for purchasing this Caps Warmer. Please read this user manual thoroughly and follow the instructions when operating the device. In addition, please keep this user manual in a safe place close to the device for future reference.

#### Intended use

This device is intended for warming composite Caps, used in dentistry, to increase flowability and improve working properties.

### 2. Before first use

Connect the base to the mains (100 - 240 VAC, 50/60 Hz) with the universal adapter. Make sure the power plug is fully seated inside the base receptacle and place the Caps tray on the base. After pressing the operating button, the indicator light on the right glows yellow to indicate that the device is connected to the mains. The base is equipped with a cover plate with non-stick coating which directs the heat to the tray.

### 3. Use

- 1.) Place the Caps you wish to warm in the recesses provided. For optimal warming, ensure that the Caps Dispenser, with the inserted, unopened Cap, is securely positioned with the Cap facing downwards (in the direction of the groove). Individual Caps warm up most efficiently when they are inserted in the recesses provided with the Cap pointing in the direction of the central axis so that the wall of the Caps has the largest possible area of contact with the wall of the recess.
- 2.) Caps Warmer has three temperature settings. Select the desired temperature by pressing the operating button repeatedly. Depending on the clinical situation and the desired consistency, the 54 °C (130 °F) or 68 °C (155 °F) temperature setting is recommended.  
The indicator light on the left shows the selected temperature as follows.

Operating button	Indicator light (left)	Temperature setting
Press once	Green	37 °C (98 °F)
Press twice	Orange	54 °C (130 °F)
Press three times	Red	68 °C (155 °F)

- 3.) After selecting the desired temperature setting, Caps Warmer needs approx. 15 - 20 minutes to heat up. During the heating process, the indicator light on the left flashes in the colour of the selected temperature setting. When it changes to a steady light, the temperature setting has been reached.
- 4.) After warming the Caps (approx. 3 minutes), apply the material within 30 seconds.
- 5.) The Caps Warmer is intended for brief warming of Caps immediately prior to application. Longer warming beyond the treatment period may damage the material and should therefore be avoided.
- 6.) To switch off the device, hold down the operating button for 2 seconds. The device automatically switches off after 10 hours if it is not used.

#### 4. Safety precautions and maintenance / disinfection

##### 1.) Safety precautions

Warning: Caps Warmer becomes hot during operation. Do not touch any hot surfaces. The Caps tray has plastic rings for handling. If the tray is removed, the base should be switched off (hold down operating button for 2 seconds) so that the cover plate cools down.

##### 2.) Maintenance / disinfection

- a) A plastic spatula should always be used to remove composite to avoid scratching the non-stick surface.
- b) Disinfection of Caps tray: The Caps tray is autoclavable, or can be wiped with a cloth and disinfectant.
- c) Disinfection of base: The base is not autoclavable. A cloth lightly moistened with disinfectant should be used to clean the plastic housing after it has cooled and while it is disconnected from the mains. Do not spray!

## 5. Troubleshooting and repair

- 1.) In the event of a malfunction, the device has a safety temperature limiter and automatically switches off as soon as the temperature exceeds the selected value. The indicator light goes out and the device cools down. Caps Warmer can be started up again after the temperature limiter resets itself.
- 2.) If it is not possible to identify the problem, please contact your local dealer or VOCO GmbH, Cuxhaven to find an optimal solution.
- 3.) Repair
  - a) Our warranty period is one year from the date of purchase. If a defect occurs within the warranty period despite the device being used properly in accordance with the user manual, you are entitled to have the device repaired free of charge (does not apply to fuses, wearing parts or power supply units).
  - b)  This symbol indicates the requirement for special disposal of electrical and electronic devices in EU countries. Do not dispose of the device as household waste. Contact your local recycling and waste disposal centre or your dealer for further information on the correct method of disposal. Please dispose of the device properly.

## 6. Explanation of symbols

Please observe the instructions for use

Recommended storage and operating temperature

Conformity with applicable European directives

Do not dispose of as household waste

You can find explanations of the symbols used at [www.voco.com/us/symbols](http://www.voco.com/us/symbols)

**7. Technical data**

Input voltage	100 - 240 VAC, 50/60 Hz
Heating time	approx. 15 - 20 minutes
Temperature settings	37 °C / 98 °F (indicator light green) 54 °C / 130 °F (indicator light orange) 68 °C / 155 °F (indicator light red)

**8. Warranty**

VOCO gives a 12-month warranty period if the device cannot be used as intended due to quality defects. This warranty begins on the day of purchase. VOCO reserves the right to either repair the device or replace individual parts or the entire device at its discretion. This warranty does not cover fuses, wearing parts or mains adapters.

**9. General information**

- We cannot accept any liability for malfunctions or damage resulting from improper use, the removal of components, modifications or inadequate maintenance.
- We cannot accept any liability for malfunctions or damage resulting from operation in unsuitable environments.
- We cannot accept any liability for malfunctions or damage resulting from acts of God.
- The warranty shall also be void in the case of product defects which can be definitively attributed to natural disasters or excessive temperatures that do not conform to the specifications in this user manual.

**10. The Caps Warmer is covered by the following patents:**

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**AdDent, Inc.**