Product description:

The set includes **Quick Up**, a gingiva-coloured, self-curing luting composite for attachments. This material is used for bonding in attachments such as ball, Locator® and telescopic anchors as well as other anchors in acrylic-based dentures (PMMA). The material can also be used for reattaching secondary elements in a denture such as bar retainers. The set also includes **Quick Up Adhesive** for permanent and firm bonding, **Quick Up LC**, a gingiva-coloured, light-curing correction material for subsequent corrections, and **Fit Test C & B** for checking the accuracy of fit and for blocking out.

Indications:

Fixing attachments such as ball, Locator® and telescopic anchors, as well as other anchors in partial and full dentures both chairside and in the laboratory. The set can also be used as repair material for reattaching secondary elements (e.g. bar retainers) in dentures.

Contraindications:

Quick Up Adhesive contains methacrylate and acetone. **Quick Up** contains benzoyl peroxide, amines, BHT and methacrylate. **Quick Up LC** contains methacrylates, BHT and amines. Do not use on patients with a known hypersensitivity (allergy) to these ingredients.

Patient target group:

Quick Up/LC and **Quick Up Adhesive** is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Quick Up/LC and **Quick Up Adhesive** should only be used by a professionally trained dental practitioner.

Please note:

Attachments and other secondary components must have mechanical retentions or, if there are no such retentions, must have been roughened mechanically, e.g. by sandblasting, before being bonded in place.

Use:

If the material is stored in the refrigerator, allow it to reach room temperature before use.

1. Preparing the denture:

Place all attachments onto the abutments and check their fit carefully. The position of the attachments can be marked on the denture with the help of articulating paper, a suitable felt-tip pen, occlusal spray, toothpaste or the like. Make oval recesses in the denture of sufficient depth to receive the attachments. The attachments must not touch the denture base. Approximately 2 mm of space must be left for **Quick Up**. Undercuts in the denture base increase the hold of the attachments. Optionally, openings can be made below the teeth on the lingual side to allow excess material to flow out and to enable a visual check to be made.

2. Checking accuracy of fit of the attachments in the denture:

The enclosed **Fit Test C & B** can be used to check whether the openings in the denture base provide enough space to receive the attachments. The instructions for use for **Fit Test C & B** should also be followed when applying the product. Fill the openings for receiving the attachments with the **Fit Test C & B** and place the denture in the patient's mouth. Ask the patient to bite down with moderate pressure and wait 2 minutes for the **Fit Test C & B** to set. Remove the denture and check the results: an even silicone layer around the attachment indicates that there is sufficient space in the denture base to receive the attachment. If there is an uneven silicone layer or if the attachment is in contact with the denture base, the openings in the denture must be enlarged and the test repeated if necessary. Remove the attachments and remove **Fit Test C & B** completely from the denture using suitable instruments (e.g. scaler or curette).

3. Insulating and blocking out:

Using Vaseline, for example, insulate all areas that are not to be permanently bonded, i.e. surfaces of dentures, abutments/implants, as well as the neighbouring teeth. To prevent accidental and irreversible adhesion of the denture to implants or abutments in the mouth, undercuts and the cervical free gingiva around the implants must be blocked out or covered with the **Fit Test C & B** or another suitable, stable material such as silicone or wax. To do this, apply **Fit Test C & B** around the undercuts and the cervical gingiva of abutments or other male parts and wait 2 minutes for it to set. Use the enclosed mixing tips type 11 to which the intraoral tip type 1 is to be attached.

4. Applying the Quick Up adhesive:

Dispense a few drops of **Quick Up Adhesive** into the mixing tray and apply to the parts of the denture to be bonded using the enclosed **Single Tim** application brush. Then dry the **Quick Up Adhesive** layer with an air syringe or dry for 30 seconds in the air. The bottle must be resealed immediately after use and stored upright.

5. Preparing the Quick Up QuickMix syringe:


Due to technical reasons there may be slight differences in the filling levels of the two syringe chambers prior to first use. Therefore remove the syringe cap and extract material until it is delivered evenly from both chambers. Afterwards place a mixing tip type 10. The notches on the syringe and the mixing tip must be lined up. The mixing tip is fixed by rotating it clockwise through 90°. The material is automatically mixed in the correct ratio in the tip as it is dispensed and can be applied immediately. All QuickMix syringes come with a detachable piston. The piston can easily be pulled backwards out of the QuickMix syringe and then reinserted. The seals in the QuickMix syringe do not change their position whilst this is being done. This prevents crosscontamination of the material in the QuickMix syringe.

The attached mixing tip is intended for single use only. After use, store the QuickMix syringe tightly sealed. Check that material flows freely through the openings before reusing.

The warranted product characteristics can only be achieved when using the supplied original mixing tips, which are also available as accessories.

6. Applying Quick Up:

Quickly fill with **Quick Up** up to a maximum of two thirds each of the prepared openings in the denture base described in points 1 and 2. You have an extraoral working time of 30 seconds. Place the denture in the patient's mouth and ask the patient to bite down with moderate pressure. Check the fit of the denture and occlusion. You have 30 seconds to check this in the mouth. After placing the denture wait 2.5 minutes before removing it from the mouth (a total time of 3.5 minutes from commencement of mixing). **Quick Up's** curing time will be extended if placed on a model for adjustments, because the prevailing temperature will be lower than that of the oral cavity. The setting time can be controlled by dispensing **Quick Up** onto a mixing pad. Higher temperatures accelerate curing. Use suitable rotary instruments if excess material has to be removed, ensuring that the attachments do not come into contact with the rotary instruments. If insufficient material has been applied or if there are areas requiring correction, use the **Quick Up LC** which is enclosed in the set (see point 7).

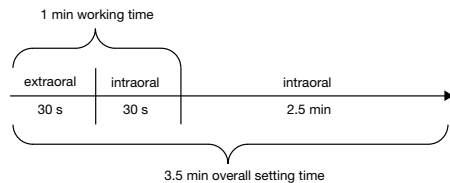7. Making corrections with Quick Up LC:

The **Quick Up LC** contained in the set is for correcting shortfalls and surface defects. To make these corrections, clean the surfaces and apply the **Quick Up Adhesive**. Then dry the **Quick Up Adhesive** layer with an air syringe or dry in the air for 30 seconds. The bottle must be resealed immediately after use and stored upright. To apply the **Quick Up LC** lock the enclosed application tip type 44 in place on the black syringe by twisting it in a clockwise direction.

The attached application tip is intended for single use only. Remove it after use and seal the syringe again tightly with its original cap.

The plunger must not be retracted during or after use, otherwise the syringe may drip or run. The material must be applied and cured in layers if corrections exceeding 1 mm in thickness are necessary. Allow at least 20 seconds curing time per layer. Conventional polymerisation devices are suitable for light-curing **Quick Up LC**. The output of the LED halogen light should be at least 500 mW/cm².

8. Finishing and final inspection:

The material used for blocking out must be removed from the abutments after bonding in the attachments. The surfaces are finished using the usual instruments (carbide bur) and polished with a flannel buffing wheel. Then place the denture in the patient's mouth and check its fit and the occlusion.

Warnings, precautionary measures:

- Phenolic substances (e.g. eugenol) impair polymerisation and therefore must not come into contact with the materials.
- Any areas of the skin that have come into contact with the products should be washed thoroughly with soap and water. In case of eye contact, rinse well with plenty of water and consult an ophthalmologist.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Storage instructions and application method:

Storage at 4°C-23°C. Always keep the **Quick Up Adhesive** bottle tightly sealed and store it upright. Do not expose the materials to direct sunlight or the operating light. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Quick Up** must be reported to VOCO GmbH and the responsible authority.

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VC 60 AA1626 E1 0222 99 © by VOCO