Rebilda Form

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Instructions for use

Rebilda Form is a shaping aid for core build-ups available in three different sizes for application with or without the simultaneous use of a root post.

Rebilda Form is white.

The shaping aids have the following advantages:

- They are transparent to the light from polymerisation lamps
- They can easily be customised and shortened by cutting
- The build-up material does not adhere to **Rebilda Form**
- Ease of removal

Application with a root post:

Firstly, select a **Rebilda Form** of suitable size, customise it to the tooth to be built up and shorten it to the required length.

If a root post is to be used, prepare the root canal as required, shorten the selected post to the required length and condition (silanise) it in accordance with the manufacturer's instructions.

Prepare the root canal and the residual tooth substance (by etching and/or priming and bonding) in accordance with the instructions for use for the products used for the adhesive technique and bond in the root post with suitable luting material (**Rebilda DC**). After polymerisation has been made (chemically or by light) in accordance with the manufacturer's instructions, place on the stump the **Rebilda Form** previously customised to the tooth to be built up and fill with composite (**Rebilda DC**). This process can be carried out in one or more steps.

Application without a root post:

Firstly, select a **Rebilda Form** of suitable size, customise it to the tooth to be built up and shorten it to the required length. Prepare the tooth substance (by etching and/or priming and bonding) in accordance with the instructions for use for the products used for the adhesive technique. After polymerising the adhesive in accordance with the manufacturer's instructions, place on the stump the **Rebilda Form** previously customised to the tooth to be built up and fill with composite (**Rebilda DC**). This process can be carried out in one or more steps.

Polymerisation:

Allow the composite used (**Rebilda DC**) to cure chemically in accordance with the manufacturer's instructions or cure with the polymerisation lamp. The material in **Rebilda Form** diminishes the performance of polymerisation lamps. You should therefore increase the curing times or, preferably, wait until the stipulated time for chemical curing has elapsed before preparation even when using a polymerisation lamp. This will ensure that the build-up material used has been completely polymerised.

Removal of Rebilda Form:

After the build-up material has been polymerised, cut the **Rebilda Form** over its entire length with a fine diamond bur with water cooling and remove. Prepare the core build-up in the usual way.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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