Perfect Bleach®

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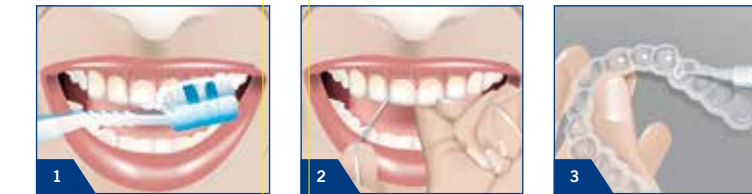
VC 60 011660 E1 0224 12

Perfect Bleach®

APPLICATION INSTRUCTION FOR THE PATIENT

Product description:

Perfect Bleach is a gentle treatment for vital and endodontically treated teeth. Perfect Bleach is used with a custom-made tray, which is fabricated by a dentist after the diagnosis has been made. The dentist will also supervise the treatment.

Discolourations can be caused by:

- age-dependent changes
- light spots
- medication (e.g. tetracycline)
- fluorosis
- food-related stains
- bleedings caused by injury of the teeth or pulp devitalisation

Perfect Bleach should not be used:

1. During pregnancy and lactation
2. On heavy smokers, unless smoking is ceased during the treatment
3. On heavy alcohol drinkers
4. On individuals who are allergic to the ingredients

Sustained damage to the enamel structure caused by the effect of carbamide peroxide-based whitening agents has not been proven.

Application:

- Brush teeth and use dental floss to clean between the teeth.
- Remove the cap from the syringe and fill the tray's hollow spaces pointing to outward with an approx. lens-sized quantity of gel (fig. 3), do not apply too much gel since this will not lead to a better result.
- The gel can also be used to partially fill the tray for severely discoloured individual teeth.
- Place the tray on the teeth and remove the excess gel from the gums with a toothbrush or finger (Fig. 5/6).
- Do not swallow the gel.
- Wear the tray on the teeth with the gel for about 2 hours. Wearing the tray overnight is also possible but not mandatory.
- Rinse the mouth and remove the remaining gel with a toothbrush.
- After wearing the tray, clean it with water and a toothbrush, allow it to dry and store it in the provided case.
- Citrus fruit, coloured fizzy drinks, red wine and nicotine should be avoided during treatment to avoid impeding the whitening process.
- Excellent oral hygiene should be maintained.

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- Do not eat or drink while wearing the tray.
- Your teeth will be much whiter after the treatment is completed, however, it is normal for the teeth to slightly darken again during the first few days after treatment. The final shade will be present approximately 2 weeks after the treatment. The treatment generally does not need to be repeated. Subsequent treatments can be carried out every 6 months, depending on the consumption of food which stains the teeth.
- Fillings and restorations do not lighten and must be replaced in some cases after the treatment. Amalgam can leave dark deposits in the tray which are harmless.

Warning:

Perfect Bleach contains 10 % / 16 % carbamide peroxide (corresponds to 3.5 % / 6 % hydrogen peroxide), potassium, menthol and fluoride. Contact with the mucosa, eyes or skin can cause irritation. If this occurs, rinse the affected areas with plenty of water. A doctor should additionally be consulted if there is contact with the eyes. Swallowing excess amounts of the gel can lead to heartburn or nausea.

The tray is made of EVA (ethylene vinyl acetate). In case of known hypersensitivity (allergies) to this ingredient, the product should not be used.

Keep out of reach of children!

Tips and safety measures:

In some cases, temporary hot / cold sensitivity may be experienced during the treatment, which usually subsides within a few days after the treatment is discontinued or ended. If sensitivity occurs, reduce the dosage and duration of the individual treatment. Check the tray for pressure points, correct fit and cracks. There is a risk of aspiration in the event of small breaks. Consult your dentist if the hypersensitivity persists. Sensitive teeth should generally only be treated with Perfect Bleach every second or third day for a maximum of two hours. Consult your dentist before using any mouth or throat medication during the treatment.

Storage:

Store the Perfect Bleach syringes in the refrigerator.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and / or written information has been given to the best of our knowledge and without obligation. Our information and / or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

Application instruction for the patient | GB

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