EN	DE	FR	ES	PT	IT
EL	NL	DA	FI	NO	SV

**VOCO**

## Provicol® QM Aesthetic

**EN** Instructions for use  
**MD** EU Medical device  
 In accordance with DIN EN ISO 3107

### Product description:

**Provicol QM Aesthetic** is an easy-to-use, eugenol-free, translucent cement for temporary luting which has no effect on the curing of composite-based restoration and luting materials. The calcium hydroxide contained in **Provicol QM Aesthetic** contributes to preserving the tooth's vitality.

Working time: ≥ 1,5 Min (23°C)  
 Setting time: ≤ 5 Min (37°C)

### Indications:

- Temporary luting of temporary restorations (crowns, bridges, inlays and onlays)
- Temporary luting of permanent restorations (crowns, bridges, inlays and onlays)
- Temporary sealing of small, single-surface cavities.

### Contraindications:

No contraindications are known for **Provicol QM Aesthetic** when the product is used correctly and as intended.

### Patient target group:

**Provicol QM Aesthetic** is suitable for application in all patients without any age or gender restrictions.

### Product performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

### Users:

**Provicol QM Aesthetic** should only be applied by a professionally trained dental practitioner.

### Use:

#### Usable mixing tips:

**VOCO** mixing tip type 10.

#### 1. Preparation

Ensure that the surfaces to be cemented are clean (please observe and comply with the Additional note, see below). Remove the protective cap from the QuickMix syringe. Prior to every use, express material until it flows evenly from both openings. Then attach a mixing tip and lock (with a 90° turn clockwise). The warranted product characteristics can only be achieved when using the supplied original mixing tips, which are also available as accessories. The attached mixing tip is intended for single use only. After use, store the QuickMix syringe tightly sealed. Check that the openings flow freely through the openings before reusing.

#### 2. Insertion of the restoration/application

Apply **Provicol QM Aesthetic** slowly into the dried restoration and onto the surface to be treated. Then place the restoration, applying gentle pressure so as to expel excess material. The working time at room temperature is at least 1.5 minutes.

#### 3. Curing

Any excess can be removed using the following methods:

- Uncured excess **Provicol QM Aesthetic** can be removed with a foam pellet or similar.
- Alternatively, wait for the material to cure (monitor in mouth) and then remove the cured excess cement with a suitable instrument.

Contact points and interproximal areas must be cleaned with dental floss. Curing is complete 5 minutes after insertion. The restoration must be held in position during curing. Assessment of the curing of **Provicol QM Aesthetic** on mixing pads or similar equipment is not reliable as the setting reactions occur differently extraorally. Complete curing following insertion of the restoration is guaranteed there at mouth temperature.

#### 4. Removal

Remove the restoration with a suitable instrument. Any remaining **Provicol QM Aesthetic** must be removed completely prior to the permanent luting. This can be done with a suitable instrument (curette, scaler, probe). A polishing paste may also be used if required.

### Warnings/precautionary measures:

- No material residue should remain in the sulcus, as this can result in irritation of the gingiva.
- In case of eye contact, rinse well with plenty of water and consult an ophthalmologist.
- Core sensitivity can be reduced by applying desensitising products prior to the temporary luting.
- Corresponding pulp protection measures are recommended if the pulp is exposed or the residual dentine thickness is insufficient.
- Temporary sealing of multiple and large cavities and temporary long-term seals are more successful with a harder cement.
- Following removal, any **Provicol QM Aesthetic** residues can be cleaned off the restorations with ease if they are firstly cooled off briefly with cooling spray extraorally.
- Our information and/or advice do not release you from the obligation of checking that the products supplied by us are suitable for their intended purpose.

### Additional note:

Moistening prepared teeth with a little water prior to the cementation facilitates subsequent removal of the restoration.

### Storage instructions and application method:

Store at 15°C to 23°C. Please store the QuickMix syringe in the sealed aluminium bag until used for the first time. After opening the bag, use the material within 6 months and before the shelf life has expired. Once opened, store protected from moisture. Do not refrigerate.

### Disposal:

Dispose of the product according to local regulations.

### Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's condition and a serious risk to public health that arise or could have arisen in association with the use of **Provicol QM Aesthetic** must be reported to VOCO GmbH and the responsible authority.

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VC 60 AA1108 E1 1122 99 © by VOCO