VOCO

Bondfix[®]

GB

CE⁰⁴⁸²

Instructions for use

Product description:

Bondfix is a single-component, light-curing, self-etch bonding agent reinforced with nanoparticles used to create a durable bond without marginal gaps between the hard substance and light-curing filling materials.

Bondfix displays the same adhesive properties as total-etch preparations without separate etching of the tooth substance.

Bondfix tolerates residual moisture.

Bondfix is an all-purpose adhesive for use with all light-curing composites, compomers and Ormocer[®]s.

Indications:

Bondfix is suited for direct restorations using light-curing filling materials in all cavity classes.

Directions:

a) Drying

A dry working area is required due to the subsequent application of a light-curing restorative material. Avoid any kind of contamination with blood or saliva.

b) Preparation

Prepare the cavity according to the principles of the adhesive restoration technique. Rinse cavity thoroughly with water. Remove excess moisture with a faint air jet.

Do not overdry dentine. The dentinal surface should remain slightly moist.

If the tooth has not been prepared (e.g. cervical defect), thoroughly clean the surface and roughen the enamel margin with a finishing diamond or etch with a phosphoric acid gel.

Protect areas close to the pulp with a suitable calcium hydroxide based liner.

c) Application

Dispense 1 drop of **Bondfix** onto a mixing palette. Apply a moderately thin layer of the adhesive to the enamel/dentine with a suitable applicator and allow it to act for 20 s.

d) Drying

Dry the adhesive layer with an air jet for at least 5 s.

e) Polymerisation

Polymerise with blue light (halogen or LED light) for 10 s.

f) Restorative

Apply the restorative according to the manufacturer’s instructions.

Warnings/precautionary measures:

Contact of the oral mucosa with **Bondfix** may result in a whitish discolouration caused by the coagulation of proteins. This is a reversible occurrence that subsides within a few days.

Phenolic substances (e.g. eugenol-containing temporary materials) interfere with polymerisation and thus must not be brought into contact with **Bondfix**.

Bondfix contains organic acids, UDMA, HEMA, camphor quinone and BHT. Do not use in cases of known allergies to any of these ingredients. Avoid contact with the oral mucosa. In case of contact of **Bondfix** with the eyes, rinse with plenty of water and seek medical advice.

Bondfix is light-curing and contains volatile ethanol. Excessive exposure to ambient light (chair light) and/or evaporation of the solvent alter the viscosity of the bonding agent. It must therefore be applied within 5 min after it has been dispensed from the bottle.

Storage and application advice:

Tightly close **Bondfix** after use. Store in an upright position to guarantee backflow of the liquid. Store at temperatures of 2 °C to 8 °C. Do not expose **Bondfix** to direct sunlight or chair light. Do not use **Bondfix** after the expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VOCO GmbH Phone **+49 (4721) 719-0**