**VOCO****Depulpin®****DE****Gebrauchsinformation**

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**VOCO****VOCO****Depulpin®****GB****Instructions for use**

**Depulpin** is a preparation for arsenic-free devitalisation of the pulp. It is fast-acting and pain-relieving. The pasty presentation makes it easy to apply.

**Composition:**

**Depulpin** contains paraformaldehyde and lidocaine hydrochloride.

**Fields of application:**

- For pulp devitalisation prior to mortal extirpation or amputation
- For residual devitalisation after removal of non-vital pulp tissue

**Dosage and application:**

In order to devitalise final teeth, expose the pulp chamber and introduce **Depulpin** paste without exerting pressure. The cavity is then to be sealed to prevent the paste from coming out or to prevent flowing out of formaldehyde. The quantity required depends on local conditions. It should be left to effect for an average of 10 - 15 days. In case of any residual pulpal activity, **Depulpin** can be reapplied after removal of the non-vital tissue.

**Contraindications:**

Allergies to formaldehyde or other ingredients.

**Side effects:**

The insertion of the material into the open pulp chamber results in more or less strong pulpitis-like complaints, which are relieved by the added local anaesthetic (lidocaine hydrochloride). In case of extirpation of the pulp, bleedings could occur at the apical extraction area in spite of the devitalisation. In case of non-sufficient diffusion or release of formaldehyde tissue residues could remain in the canal which cause severe pain. If formaldehyde should leave the apex, inflammations resp. necrosis could occur in the periapical tissue. If applied to primary dentition, in very rare cases the tooth bud of the following definite tooth could be damaged in the early stage of development (before end of mineralisation).

**Storage:**

**Depulpin** paste should be kept tightly closed and not be exposed to light or heat. Store at temperature max. 25 °C. Do not use after expiry date.

Last revised: 04/2017

VOCO GmbH Phone +49 (4721) 719-0  
Anton-Flettner-Str. 1-3 Fax +49 (4721) 719-140  
27472 Cuxhaven e-mail: marketing@voco.com  
Germany www.voco.com

**VOCO**