



# V-Print dentbase

## Indikationen / Indications

Herausnehmbare Prothesenbasen / Removable denture bases

Druckerhersteller / manufacturer	3D-Druck / 3D printing			Reinigung / Washing		Nachbelichtung / Postcuring			
	Gerät / Device	Schicht / Layer* [µm]	Build Style*	Gerät / Device	Programm / Program	Gerät / Device	Programm / Program		
VOCO	SolFlex 170 [SMP], SolFlex 350 [SMP], SolFlex 650 [SMP], SolFlex 170 HD	50	Dental Gentle	Ultraschall / Ultra sonic	3 min. / 2 min.	OtoFlash G171	2 × 2000 flashes		
				Rührbad / stirring bath	3 min. / 2 min.	Next Dent LC Box	1 × 30 min.		
						Hilite power 3D	2 × 5 min.		
						OtoFlash G171	2 × 2000 flashes		
						Next Dent LC Box	1 × 30 min.		
				Hilite power 3D	2 × 5 min.	OtoFlash G171	2 × 2000 flashes		
Next Dent LC Box	1 × 30 min.								
W2P	SolFlex 170 [Plus] SolFlex 350 [Plus] SolFlex 650 [Plus] SolFlex 163 FullHD	50	Dental Gentle	Ultraschall / Ultra sonic	3 min. / 2 min.	OtoFlash G171	2 × 2000 flashes		
				Rührbad / stirring bath	3 min. / 2 min.	Next Dent LC Box	1 × 30 min.		
						Hilite power 3D	2 × 5 min.		
						OtoFlash G171	2 × 2000 flashes		
						Next Dent LC Box	1 × 30 min.		
				Hilite power 3D	2 × 5 min.	OtoFlash G171	2 × 2000 flashes		
Next Dent LC Box	1 × 30 min.								
ASIGA	MAX UV PRO 4K	50	×	Ultraschall / Ultra sonic	3 min. / 2 min.	OtoFlash G171	2 × 2000 flashes		
				Rührbad / stirring bath	3 min. / 2 min.	Next Dent LC Box	1 × 30 min.		
						Imprimo Cure	1 × 10 min.		
						OtoFlash G171	2 × 2000 flashes		
						Next Dent LC Box	1 × 30 min.		
				Imprimo Cure	1 × 10 min.	OtoFlash G171	2 × 2000 flashes		
Next Dent LC Box	1 × 30 min.								
Rapid Shape	D20 II D30 II D40 II D10+ D20+ D30+ D50+	100	×	Ultraschall / Ultra sonic	3 min. / 2 min.	OtoFlash G171	2 × 2000 flashes		
				Rührbad / stirring bath	3 min. / 2 min.	Next Dent LC Box	1 × 30 min.		
						RS Wash	V-Print dentbase	RS Cure / RS Cure US	V-Print dentbase
						OtoFlash G171	2 × 2000 flashes		
						Next Dent LC Box	1 × 30 min.		
				OtoFlash G171	2 × 2000 flashes	OtoFlash G171	2 × 2000 flashes		
Next Dent LC Box	1 × 30 min.								
DMG	3Delite 3Demax	100	×	Ultraschall / Ultra sonic	3 min. / 2 min.	OtoFlash G171	2 × 2000 flashes		
				Rührbad / stirring bath	3 min. / 2 min.	Next Dent LC Box	1 × 30 min.		
						DMG 3Dewash	V-Print dentbase	3Decure	V-Print dentbase
						OtoFlash G171	2 × 2000 flashes		
						Next Dent LC Box	1 × 30 min.		
				OtoFlash G171	2 × 2000 flashes	OtoFlash G171	2 × 2000 flashes		
Next Dent LC Box	1 × 30 min.								
Sera	SHERAprint 20 SHERAprint 30 SHERAprint 40 SHERAprint 20+ SHERAprint 30+ SHERAprint 50+	100	×	Ultraschall / Ultra sonic	3 min. / 2 min.	OtoFlash G171	2 × 2000 flashes		
				Rührbad / stirring bath	3 min. / 2 min.	Next Dent LC Box	1 × 30 min.		
						SHERAprint-wash	V-Print dentbase	SHERAprint-cure	V-Print dentbase
						OtoFlash G171	2 × 2000 flashes		
						Next Dent LC Box	1 × 30 min.		
				OtoFlash G171	2 × 2000 flashes	OtoFlash G171	2 × 2000 flashes		
Next Dent LC Box	1 × 30 min.								
Straumann	P20 II P30 II P40 II P10+ P20+ P30+ P50+	100	×	Ultraschall / Ultra sonic	3 min. / 2 min.	OtoFlash G171	2 × 2000 flashes		
				Rührbad / stirring bath	3 min. / 2 min.	Next Dent LC Box	1 × 30 min.		
						P wash	V-Print dentbase	P cure / P cure US	V-Print dentbase
						OtoFlash G171	2 × 2000 flashes		
						Next Dent LC Box	1 × 30 min.		
				OtoFlash G171	2 × 2000 flashes	OtoFlash G171	2 × 2000 flashes		
Next Dent LC Box	1 × 30 min.								
Dentalwings	D20 II D30 II D40 II D10+ D20+ D30+ D50+	100	×	Ultraschall / Ultra sonic	5 min. / 3 min.	OtoFlash G171	2 × 2000 flashes		
				Rührbad / stirring bath	5 min. / 3 min.	Next Dent LC Box	1 × 25 min.		
						OtoFlash G171	2 × 2000 flashes		
				Next Dent LC Box	1 × 25 min.				
OtoFlash G171	2 × 2000 flashes	OtoFlash G171	2 × 2000 flashes						
		Next Dent LC Box	1 × 25 min.						
Microlay	Versus	50	×	Ultraschall / Ultra sonic	3 min. / 2 min.	OtoFlash G171	2 × 2000 flashes		
				Rührbad / stirring bath	3 min. / 2 min.	OtoFlash G171	2 × 2000 flashes		

× = Nicht anwendbar / not applicable \*empfohlene/r Schichtstärke/Build Style / recommended layer thickness/build style

VOCO GmbH  
Anton-Flettner-Str. 1-3  
27472 Cuxhaven  
Germany

Phone +49 (4721) 719-0  
Fax +49 (4721) 719-140  
e-mail: [marketing@voco.com](mailto:marketing@voco.com)  
[www.voco.dental](http://www.voco.dental)

**VOCO**  
THE DENTALISTS