EN Instructions for use

MD EU Medical device

Product description:

CediTEC is a luting agent system for the permanent luting of methacrylate-based prosthetic teeth in methacrylate-based denture bases. The **CediTEC Adhesive cartridge** mixes the paste automatically without producing any errors or bubbles. The system comprises **CediTEC Adhesive (cartridge)** and **CediTEC Primer**.

Colour:
Pink

Indications:

- Luting of prefabricated prosthetic teeth and CAD/CAM-produced individual teeth and tooth sections in CAD/CAM-produced denture bases
- Luting of prosthetic teeth in the scope of a repair or extension

Contraindications:

- CediTEC contains methacrylates, benzoyl peroxide, butylated hydroxytoluene (cartridge), methacrylates, acetone (Primer). CediTEC should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.
- Luting of ceramic teeth

Patient target group:

CediTEC is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

CediTEC should only be used by a professionally trained dental practitioner.

Use:

Preparation:

Checking the accuracy of fit:

Check the prosthetic teeth's accuracy of fit before sticking them in position in the denture base.

Preparation of the prosthetic teeth:

For an optimal bond, blast the luting surface of the prosthetic teeth with aluminium oxide (50 µm-110 µm, 1 bar-2 bar). Use a suction device to remove any dust.

Remove abrasive material residues carefully with an ultrasound bath, steam cleaner or compressed air. The instructions for use from the manufacturers must be observed and followed.

Note: Observe and follow any further information provided by the manufacturer regarding the pre-treatment of the prosthetic teeth!

Preparation of the denture base:

For an optimal bond, blast the luting surface of the denture base with aluminium oxide (50 µm-110 µm, 1 bar-2 bar). Use a suction device to remove any dust.

Remove abrasive material residues carefully with an ultrasound bath, steam cleaner or compressed air. The instructions for use from the manufacturers must be observed and followed.

Note: Observe and follow any further information provided by the manufacturer regarding the pre-treatment of the denture base!

Luting:

1. Applying the primer

Pour a few drops of **CediTEC Primer** onto a mixing tray and apply it to the points to be stuck together using the enclosed Easy Brush application brush. The bottle must be resealed immediately after use.

Prosthetic teeth

Apply the primer to the adhesive surfaces of the teeth and allow to dry for 30 seconds.

Denture base

Apply the primer generously to the adhesive surfaces of the cavities and allow to dry for 30 seconds.

2. Preparation of the cartridge

Insert the **CediTEC Adhesive cartridge** in the **VOCO Dispenser** (type 2) or a similar mixing gun with a suitable plunger. For technical reasons there may be slight differences in the fill levels of both cartridges chambers.

You should therefore remove the cartridge cap and continue to press out the material until an equal amount has been ejected from both outlets. Then attach a mixing tip (type 20) and lock in place with a 90° turn. The material is expelled by pressing the lever and is automatically mixed in the correct ratio.

Refit the original twist cap to prevent the **CediTEC Adhesive** pastes drying out during storage.

3. Application of CediTEC Adhesive

Apply **CediTEC Adhesive** to the tooth cavities and insert individual teeth and/or tooth sections directly. The quantity of luting agent is dependent upon the size of the cavities.

Large quantities of excess material should be removed using a suitable instrument directly after insertion of the prosthetic teeth. Smaller quantities of excess material can be removed with finishing diamonds or flexible polishing wheels following the polymerisation.

Note: The working time as of the application is 4 minutes (at 23 °C).

4. Polymerisation in the pressure pot

To achieve an optimal adhesive bond, the polymerisation must occur in the pressure pot immediately after insertion of the teeth or removal of the excess material.

Curing time: 15 minutes

Water temperature: 50 °C

Pressure: 2-6 bar

Warnings/precautionary measures:

Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

- Store the prime bottle sealed properly at all times.
- Avoid skin contact with the primer. Any areas of the skin that have come into contact with the products should be washed thoroughly with soap and water.
- In case of eye contact, rinse well with plenty of water and consult an ophthalmologist.
- The warranted product characteristics can only be achieved when using the supplied original mixing tips, which are also available as accessories.

Storage instructions and application method:

Store at between 4 °C and 23 °C. If the material is stored in the refrigerator, bring it to room temperature before use. Do not store the product in a place where it is exposed to direct artificial light or sunlight. Do not use the product after its expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **CediTEC** must be reported to VOCO GmbH and the responsible authority.

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