# VOCO

## Meron Plus AC

**EN** Instructions for use  
**MD** EU Medical device  
 In accordance with DIN EN ISO 9917-2, class 1

### Product description:

Meron Plus AC is a radiopaque, chemically curing, resin-modified glass ionomer luting cement in easy-to-use application capsules. The characteristics of the material are easy handling, low solubility in the mouth and high self-adhesion. Each capsule contains a minimum of 0.14 ml of usable cement.

### Indications:

- Luting of
  - porcelain-fused-to-metal crowns, bridges on hard dental tissue and on cores rebuilt with amalgam, composites or glass ionomer materials
  - metal inlays, onlays, crowns and bridges
  - root posts
  - all-ceramic crowns manufactured from silicate ceramic, zirconium dioxide-only or alumina-only ceramic cores
  - orthodontic appliances

### Contraindications:

Meron Plus AC contains polyacrylic acid, peroxide and methacrylates. Meron Plus AC should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

### Patient target group:

Meron Plus AC is suitable for use in all patients without any age or gender restrictions.

### Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

### Use:

Meron Plus AC should only be used by a professionally trained dental practitioner.

### Use:

**Preparation:**  
Thoroughly clean and dry enamel, dentine and cavity surfaces. Clean cores before luting (e.g. pumice, water), thoroughly rinse and dry with a gentle stream of air or cotton pellets (do not overdry!).  
Before application, the capsule has to be activated, mixed and opened.

### Activating the application capsule and mixing the material:

Only extract the application capsule from the tubular bag immediately before use. During activation, the liquid from the plunger must be pressed into the inside of the capsule as fully as possible. To achieve this, the end of application capsule where the plunger of liquid protrudes is slowly pressed down onto a hard surface (e.g. a table) as far as the plunger will move (see fig. 1). The capsule must only be activated directly before use and loaded immediately into a high-frequency mixer with at least 4,000 oscillations per minute. Mix the activated application capsule for 10 s, remove it from the mixer without delay and insert it in the holder of an application device (e.g. AC Applicator, see corresponding instructions for use).

### Application:

Open the outlet by lifting the cannula until you feel a stop (see fig. 2), and apply the material within 10 s of the end of mixing by compressing the lever on the applicator slowly and evenly. The cannula can be aligned as required by turning the whole application capsule inside the holder. Working time and consistency may be influenced by the respective type of capsule mixer and mixing time.

### Luting of restorative pieces:

Apply the material onto the prepared area and/or into the restorative piece. Then place the restorative piece *in situ* and let it set under slight pressure.

Any unnecessary waiting times between activation, mixing and application should be avoided, as the process of curing may impede extraction of the material and positioning of the restoration.

Meron Plus AC has a working time of approx. 2 min at room temperature and sets in the mouth in 3 - 5 min. After approx. 4 min the material reaches a gel-like consistency, making it easy to remove any excess material with the aid of a probe. Higher temperatures shorten and lower temperatures extend these working times.

### Luting of posts:

Apply Meron Plus AC only to the post and then place the post directly into the root canal. Vibrate the post slightly during placement to avoid trapping air. Do not apply the cement into the canal with a lentulo since this could increase the speed of setting.

Usable quantity per capsule	min. 0.14 ml
Net weight per capsule	min. 0.27 g
Mixing time	10 s
Working time	approx. 2 min
Setting time	3 - 5 min

### Warnings, precautionary measures:

- Ensure that the application capsule is mixed immediately after activation.
- Phenolic substances, especially preparations containing eugenol or thymol, inhibit the curing of Meron Plus AC. Therefore, do not use zinc oxide eugenol cements or other eugenol-containing substances in combination with Meron Plus AC.
- Rinse instruments immediately with cold water before the cement residues have set. After removal of orthodontic bands, remove cement residues quickly and gently with an ultrasonic scaler.

- Avoid contact of Meron Plus AC with soft tissue.
- In case of eye contact, rinse well with plenty of water and consult an ophthalmologist.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

### Constituents (in descending order according to content):

Fluorosilicate glass, HEMA, water, polyacrylic acid, GlyDMA, tartaric acid, UDMA, initiators, stabilisers, colour pigments

### Storage instructions and application method:

Store Meron Plus AC in a dry place at temperatures of 15 °C to 23 °C. Only remove it from the foil immediately before use. Do not use after the expiry date.

### Disposal:

Dispose of the product in accordance with local regulations.

### Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of Meron Plus AC must be reported to VOCO GmbH and the responsible authority.

### Note:

The Summary of Safety and Clinical Performance of Meron Plus AC can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>). Detailed information can also be found at [www.voco.dental](http://www.voco.dental).

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VC 60 BB1736 E1 0225 99 © by VOCCO