



V-Print c&b temp

Indikationen / Indications
Hochästhetischer Provisorien/-Langzeitprovisorien

| Druckerhersteller / manufacturer | 3D-Druck / 3D printing | | Reinigung / Washing | | Nachbelichtung / Postcuring | | |
|----------------------------------|---|--------------------------|---------------------|----------------|--|-------------------------------------|--------------------|
| | Gerät / Device | Schicht / Layer* [µm] | Build Style* | Gerät / Device | Programm / Program | Gerät / Device | Programm / Program |
| VOCO ¹ | SolFlex 170 [SMP] SolFlex 350 [SMP] SolFlex 650 [SMP] SolFlex 170 HD | 100 | Gentle | – | Pinselreinigung / cleaning with brush | OtoFlash G171 (280N2 Schale / tray) | 2 x 2000 flashes |
| W2P ¹ | SolFlex 170 [Plus] SolFlex 350 [Plus] SolFlex 650 [Plus] SolFlex 163 FullHD SolFlex 250 UHD | 100 | Gentle | – | Pinselreinigung / cleaning with brush | OtoFlash G171 (280N2 Schale / tray) | 2 x 2000 flashes |
| ASIGA | MAX UV PRO 4K | 100 | x | – | Pinselreinigung / cleaning with brush | OtoFlash G171 (280N2 Schale / tray) | 2 x 2000 flashes |
| Rapid Shape | D20 II D30 II D40 II D10+ D20+ D30+ D50+ | 50 | x | – | Pinselreinigung / cleaning with brush | OtoFlash G171 (280N2 Schale / tray) | 2 x 2000 flashes |
| DMG | 3Delite 3Demax | 50 | x | – | Pinselreinigung / cleaning with brush | OtoFlash G171 (280N2 Schale / tray) | 2 x 2000 flashes |
| Shera | SHERAprint 20 SHERAprint 30 SHERAprint 40 SHERAprint 20+ SHERAprint 30+ SHERAprint 50+ | 50 | x | – | Pinselreinigung / cleaning with brush | OtoFlash G171 (280N2 Schale / tray) | 2 x 2000 flashes |
| Straumann | P20 II P30 II P40 II P10+ P20+ P30+ P50+ | 50 | x | – | Pinselreinigung / cleaning with brush | OtoFlash G171 (280N2 Schale / tray) | 2 x 2000 flashes |
| Dentalwings | D20 II D30 II D40 II D10+ D20+ D30+ D50+ | 50 | x | – | Pinselreinigung / cleaning with brush | OtoFlash G171 (280N2 Schale / tray) | 2 x 2000 flashes |
| Microloy | Versus | 100 | x | – | Pinselreinigung / cleaning with brush | OtoFlash G171 (280N2 Schale / tray) | 2 x 2000 flashes |
| MiiCraft | Prime 150Y Ultra 125Y / 150Y Hyper 125Y Profession 250Y | 100 | x | – | Pinselreinigung / cleaning with brush | OtoFlash G171 (280N2 Schale / tray) | 2 x 2000 flashes |
| Ackuretta | SOL | Fine | x | – | Pinselreinigung / cleaning with brush | OtoFlash G171 (280N2 Schale / tray) | 2 x 2000 flashes |
| Ackuretta | SOL Plus | Standard | x | – | Pinselreinigung / cleaning with brush | OtoFlash G171 (280N2 Schale / tray) | 2 x 2000 flashes |
| Shining 3D | AccuFab-L4D | 100 | x | – | Pinselreinigung / cleaning with brush | OtoFlash G171 (280N2 Schale / tray) | 2 x 2000 flashes |
| UNIZ Technology | NBEE | 100 | x | – | Pinselreinigung / cleaning with brush | OtoFlash G171 (280N2 Schale / tray) | 2 x 2000 flashes |
| UnionTech | E128 E230 | 100 | x | – | Pinselreinigung / cleaning with brush | OtoFlash G171 (280N2 Schale / tray) | 2 x 2000 flashes |
| DENTAL MAKERS | LightBuilder 4K | 100 | x | – | Pinselreinigung / cleaning with brush | OtoFlash G171 (280N2 Schale / tray) | 2 x 2000 flashes |

x = Nicht anwendbar / not applicable *empfohlene/r Schichtstärke/Build Style / recommended layer thickness/build style

¹ nur mit PowerVat / only with PowerVat

VOCO GmbH
Anton-Flettner-Str. 1-3
27472 Cuxhaven
Germany

Phone +49 (4721) 719-0
Fax +49 (4721) 719-140
e-mail: marketing@voco.com
www.voco.dental

