VOCO Admira® Protect

EN Instructions for use
MD EU Medical device

Product description:

Light-curing, fluoride-releasing ORMOCKER®-based preparation for the treatment of dentine hypersensitivity.

Admira Protect completely reduces hypersensitivity, in most cases after only one application, and additionally protects against external influences and abrasion.

Indications:

- Treatment of hypersensitive dentine
- Treatment of cervical areas (including crown margins, after tooth cleaning or tartar removal, exposed cervical areas following surgery etc.)

Contraindications:

Admira Protect contains methacrylate, BHT and organic acids. **Admira Protect** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

Patient target group:

Admira Protect is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Admira Protect should only be used by a professionally trained dental practitioner.

Use:

1. Preparation

Clean the teeth to be treated.
Isolate areas to be treated with cotton rolls. Remove excess moisture with an airjet. Do not overdry dentine. The dentine surface should be slightly moist.
Avoid any contamination with blood or saliva after cleaning.

2. Dispensing

Dispense **Admira Protect** onto a **Single Tim** or a disposable brush. Alternatively, dispense **Admira Protect** onto a mixing pallet.

3. Application

Apply **Admira Protect** with **Single Tim** or a disposable brush evenly on all dentine surfaces to be treated and allow to act for 20 s. Then disperse **Admira Protect** with a faint airjet and light-cure with a conventional polymerisation device for 10 s. The light output should be a minimum of 500 mW/cm². Apply a second layer of **Admira Protect**, disperse it with a faint airjet and light-cure for 10 s.

4. Removal of the inhibition layer

Remove the oxygen-inhibited layer (soft surface) with a cotton pellet or a cotton roll. Check the gingival margins and, if necessary, remove any residual material.

Warnings, precautionary measures:

- Phenolic substances (e.g. eugenol-containing materials) disturb polymerisation and must not be brought into contact with **Admira Protect**.
- Avoid contact with the mucosa.
- In case of eye contact, rinse well with plenty of water and consult an ophthalmologist.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Constituents (in descending order according to content):

Acetone, BisGMA, methacrylate phosphoric acid ester, HEMA, UDMA, TEGDMA, methacrylate functionalised polyacid, ORMOCKER® resin, initiators, stabilisers

Storage instructions and application method:

Close the **Admira Protect** bottle tightly after use. Store at temperatures of 4 °C to 23 °C (storage at 5 °C - 8 °C prolongs the shelflife of the preparation). Do not expose **Admira Protect** to direct sunlight or the chair light.
Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Admira Protect** must be reported to VOCO GmbH and the responsible authority.

Note:

The Summary of Safety and Clinical Performance of **Admira Protect** can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

Detailed information can also be found at www.voco.dental.

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ORMOCER® is a registered trademark of Fraunhofer Gesellschaft zur Förderung der angewandten Forschung e.V., München, Germany.