EN Instructions for use
MD EU Medical device

Product description:
Calcicur is a radiopaque, ready-to-use water-based calcium hydroxide paste with a pH value of >12.5.

Indications:

- Indirect capping for the treatment of caries profunda
- Direct capping when the pulp is opened or a pulpotomy is performed
- Lining of cavities to prevent exposure to acid media when using cements
- For temporary filling of root canals

Contraindications:

Calcicur is contraindicated in patients with a known sensitivity or allergy to device chemical material.

Patient target group:

Calcicur is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Calcicur should only be used by a professionally trained dental practitioner.

Use:

Clean and dry the cavity before applying **Calcicur** paste in thin layers until the desired thickness is reached. Allow each layer to dry briefly or dry carefully with an air syringe. Do not apply paste to the cavity margins and remove any material adhering to them. Then place lining and filling.

For root canal fillings, first prepare and clean the root canal (e.g. hydrogen peroxide 3 % and sodium hypochlorite solution) and then insert paste into the root canal with the enclosed application cannula or with a Lentulo. Then move the paste towards the tip of the root using the Lentulo.

Take care to avoid trapping air or pushing the material beyond the apex, then place the filling. Checking the temporary root canal filling by x-ray is recommended. Renew the temporary root canal filling after extended retention time (more than 6 - 8 months).

Temporary filling of the root canal with **Calcicur** can be carried out either before or after complete root maturation.

The attached application tip is intended for single use only. Remove it after use and seal the syringe again tightly with its original cap. The use of a protective barrier is recommended to protect the syringe against contamination with bodily fluids, dirt from hands or oral tissue.

Warnings, precautionary measures:

- Improper storage can cause drying out or separating of the paste.
- Calcium hydroxide is caustic to skin and mucous membranes. Avoid contact with the eyes! In the event of accidental contact, rinse thoroughly with water and seek medical advice if necessary.
- Those areas of the hard tooth substance which will be adhesively bonded to dental materials must not be treated with calcium hydroxide.
- Do not apply **Calcicur** in thick layers or as a permanent filling or luting material.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Constituents (in descending order according to content):

Water, calcium hydroxide, barium sulfate, hydroxyethylcellulose

Storage instructions and application method:

Do not use hardened paste. Storage at 15°C-23°C. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Calcicur** must be reported to VOCO GmbH and the responsible authority.

Note:

The Summary of Safety and Clinical Performance of **Calcicur** can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>). Detailed information can also be found at www.voco.dental.

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VC 60 AA1093 E1 1022 99 © by VOCO