VOCO

Grandio® Core Dual Cure

EN Instructions for use
MD EU Medical device

In accordance with DIN EN ISO 4049

Product description:

Grandio Core Dual Cure is a dual-curing, radiopaque, flowable composite for core build-up, with very good mechanical properties.

Grandio Core Dual Cure contains 77 % w/w inorganic fillers in a methacrylate matrix.

Shades:

Grandio Core Dual Cure is available in three different shades:

- dentine shade for high aesthetic demands, e.g. under all-ceramic restorations or highly translucent composite restorations
- blue to indicate the transition between the material and the dental tissue, as a base for metal crowns or faced crowns with a metal base
- white as an option for indicating preparation borders, whilst simultaneously maintaining aesthetic requirements

Indications:

- Core build-up of vital or non-vital teeth
- Luting of root posts

Contraindications:

Grandio Core Dual Cure contains methacrylates, amines, benzoyl peroxide and BHT. **Grandio Core Dual Cure** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

Patient target group:

Grandio Core Dual Cure is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Grandio Core Dual Cure should only be used by a professionally trained dental practitioner.

Use:

Preparation:
Bring material to room temperature before use.

Moisture control:

Ensure that the working field is sufficiently dry. The use of a rubber dam is recommended.

Cavity preparation:

In principle, the cavities should be prepared according to the rules for adhesive restorative treatment, i.e. preparation should be minimally invasive in order to protect healthy dental hard tissue. Then clean and dry the cavity.

Lining:

A suitable pulp-protective liner that can be covered with a stable cement, if necessary, should be applied in close proximity to the pulp.

Bonding materials:

Grandio Core Dual Cure is applied with a dentine/enamel bond indicated for dual-curing composites, using the adhesive technique. Please follow the respective instructions for use in regard to preparation and application.

Preparing the QuickMix syringe:

For technical reasons, there may be slight differences in the filling levels of the two syringe chambers prior to initial use. Therefore remove the syringe cap and continue to extrude material until it flows evenly from both outlets. Then attach a type 11 mixing tip. The notches on the syringe and the mixing tip must be lined up. The mixing tip is fixed by rotating clockwise through 90°. If necessary, an additional intraoral tip type 4 can be placed on the end of the mixing tip. The material is automatically mixed in the correct proportions inside the mixing tip as it is dispensed and can be applied immediately.

The attached mixing tip is intended for single use only. After use, store the QM syringe tightly sealed. Check that material flows freely through the openings before reusing.

The warranted product characteristics can only be achieved when using the supplied original mixing tips, which are also available as accessories.

Application of core build-up materials:

Place matrix around the prepared tooth or apply shaping aid and **Grandio Core Dual Cure** directly from the QuickMix syringe. **Grandio Core Dual Cure** has a working time of approx. 1.5 min (23 °C).

Light-curing:

Apply mixed material into the cavity in increments of no more than 2 mm and polymerise for 40 s each time using an LED / halogen lamp with a minimum light output of 500 mW/cm². Hold the light emitting window of the polymerisation lamp as close to the surface of the core build-up as possible, as curing may otherwise be insufficient. The restoration can be finished immediately after light-curing.

Chemical curing:

If light-curing is not carried out, the build-up can be processed 5 min after insertion into the mouth at the earliest.

Luting of root posts:

Use an adhesive indicated for dual-curing composites for the luting of root posts.

Please follow the respective instructions for use in regard to preparation and application.

1. Using the QuickMix syringe:

It is recommended that **Grandio Core Dual Cure** is applied with the fine intraoral tip type 1 directly into the root canal. Do not use Lentulos! In addition, **Grandio Core Dual Cure** can be applied to wet the root post, when the latter has been prepared according to the manufacturer's instructions. Insert the post making sure to leave a slight excess of cement. Due to the increased temperature and the contact with the bonding agent, the working time of **Grandio Core Dual Cure** in the root canal is reduced to 40 s.

2. To fix the root post and to cure the occlusally applied bonding, the material should be light-cured for at least 40 s.

3. For core build-up, proceed as described in "Application of core build-up materials".

Warnings, precautionary measures:

- Phenolic substances, especially preparations containing eugenol and thymol, lead to curing failures in composites. The use of zinc oxide eugenol cements or other eugenol-containing materials in combination with composites is therefore to be avoided.
- The use of one-bottle adhesive systems is to be avoided, as an optimal bond with chemically or dual-cured materials cannot, as a general rule, be guaranteed.
- Isolate the core build-up for taking impressions or fabricating temporary crowns/bridges (it may be sufficient to moisten it thoroughly), in order to prevent any bonding with these materials.
- Avoid contact with the oral mucosa.
- In cases of purely chemical curing, low temperatures (e.g., when working on the anterior region) can lead to an increase in setting time.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Constituents (in descending order according to content):

Barium aluminium borosilicate glass, TEGDMA, ytterbium trifluoride, silicon dioxide, UDMA, BisGMA, fumed silica, initiators, stabilisers, pigments

Storage:

Do not expose **Grandio Core Dual Cure** to direct sunlight or the operating light. Storage at 4°C-23°C. Refrigeration is recommended. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Grandio Core Dual Cure** must be reported to VOCO GmbH and the responsible authority.

Note:

The Summary of Safety and Clinical Performance of **Grandio Core Dual Cure** can be found in the European database on medical devices (EUDAMED - <https://ec.europa.eu/tools/eudamed>). Detailed information can also be found at www.voco.dental.

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VC 60 AA1910 E1 0422 99 © by VOCO