VOCO

DE

Remin Pro® forte

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USA

Product description:

Remin Pro forte is a water-based cream that contains hydroxyapatite, sodium fluoride (1,450 ppm fluoride) and xylitol. **Remin Pro forte** provides extra protection for teeth and, in doing so, it helps neutralise acids from acid-forming bacteria in plaque. **Remin Pro forte** additionally aids the neutralisation of other acids in the mouth.

In addition, **Remin Pro forte** contains extracts of ginger (*Zingiber officinale*) and curcuma (*Curcuma xanthorrhiza*).

Indications:

- After tooth whitening
- After professional tooth cleaning
- For the prevention and control of hypersensitivities
- During orthodontic treatment

Application:

1. Apply a bean-sized amount of **Remin Pro forte** to the teeth with a finger or suitable instrument (toothbrush or swab) and distribute.
2. The patient should distribute the remaining **Remin Pro forte** cream in his or her mouth with the tongue. **Remin Pro forte** and the saliva should remain in the mouth for as long as possible (minimum of 3 minutes) for an optimal result.
3. The patient should spit out the remaining amount. Rinsing should be avoided if possible. The patient should wait at least 30 minutes after the treatment before eating or drinking.

Also for daily use after oral hygiene.

Storage:

Store **Remin Pro forte** in a cool, dry area protected from direct sunlight (4 °C - 23 °C / 39 °F - 73 °F). Do not use the material after the expiry date.

Precautions:

1. Always close the tube immediately after use.
2. The application of **Remin Pro forte** is not recommended for patients under 6 years or for home use on children under 12 years.
3. In case of contact with the eyes, flush thoroughly with plenty of water and seek medical attention.

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith. Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

Keep this material out of reach of children.

For dental use only.

An explanation of the symbols used in labeling can be found at www.voco.com/symbols

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Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

Last revised: 12/2016

VOCO GmbH Phone +49 (4721) 719-0
Anton-Flettner-Str. 1-3 Fax +49 (4721) 719-140
27472 Cuxhaven e-mail: marketing@voco.com
Germany www.voco.com