EN	DE	FR	ES	IT
NL	DK	SE	NO	SV

**VOCO**

**Traypurol**

**EN MD** EU Medical device

**Concentrated cleaning agent for impression trays and instruments**

**Traypurol** removes residues of alginates, zinc oxide eugenol, carboxylate and phosphate cement from impression trays and instruments without using ultrasonic apparatus.

**Traypurol** is biodegradable and provides gentle cleaning action.

**Instructions for use:**

**Performance features:**

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

**User:**

**Traypurol** should only be applied by a professionally trained dental practitioner.

Use **Traypurol** in a 5 to 10 % solution (dilute approx. 50 - 100 ml **Traypurol** in 1 l water). For persistent residues, **Traypurol** can also be used in a more concentrated solution. Warm water accelerates the cleaning process.

Pre-clean the trays and instruments and place them in the **Traypurol** solution. An upright position of the impression trays in the solution will enhance the cleaning process.

Residues are softened in the **Traypurol** solution within a few hours and can then be easily rinsed off under running water or removed with a brush. If possible leave impression trays and instruments soaking overnight in the solution.

**Traypurol** is suitable for all non-corroding metals and plastics. Renew the solution at least after 2 - 3 days. **Traypurol** is also very suitable for use in ultrasonic apparatus.

**Warnings/precautionary measures:**

- Safety glasses and gloves should be worn while using this product.
- In case of eye contact, rinse well with plenty of water and consult an ophthalmologist.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

**Storage:**

Store at temperatures of 4 °C to 23 °C. Do not use after expiry date.

**Disposal :**

Dispose of the product according to local regulations.

**Reporting obligation:**

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's condition and a serious risk to public health that arise or could have arisen in association with the use of **Traypurol** must be reported to VOCO GmbH and the responsible authority.

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VOCO GmbH Phone +49 (4721) 719-0