# VOCO Profluorid® Varnish

**EN** Instructions for use  
MD EU Medical device

## Varnish with fluoride for desensitizing teeth

### Product description:

**VOCO Profluorid Varnish** is a fluoride-containing varnish for tooth desensitisation used in the treatment of enamel and dentine surfaces. **VOCO Profluorid Varnish** also adheres to moist surfaces and tolerates residual moisture and saliva.

**VOCO Profluorid Varnish** is an ethanolic suspension on the basis of colophony with artificial flavouring and xylitol as a sweetener. 1 ml of **VOCO Profluorid Varnish** contains 50 mg sodium fluoride, which corresponds to 22,6 mg of fluoride.

### Indications:

**VOCO Profluorid Varnish** is indicated for:

1. the treatment of hypersensitive teeth;
2. the treatment of sensitive root surfaces

### Contraindications:

- **VOCO Profluorid Varnish** contains colophony and artificial flavouring. **VOCO Profluorid Varnish** should not be used in patients with a known hypersensitivity (allergy) to these constituents.
- **VOCO Profluorid Varnish** should not be used in patients suffering from ulcerative gingivitis or ulcerative oral mucositis.
- **VOCO Profluorid Varnish** is not indicated for systemic treatment.

### Patient target group:

**VOCO Profluorid Varnish** is suitable for use in all patients without any age or gender restrictions.

### Product performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

### User:

**VOCO Profluorid Varnish** should only be applied by a professionally trained dental practitioner.

### Use:

#### Recommended dose:

- Treatment of patients with deciduous teeth – approx. 0.25 ml of **VOCO Profluorid Varnish**
- Treatment of patients with mixed or permanent teeth – approx. 0.40 ml of **VOCO Profluorid Varnish**

Apply a thin film of **VOCO Profluorid Varnish** to the dental hard tissue. A thick film detaches more easily and offers no therapeutic advantages. Based on experience, the required quantity for an entire set of teeth is between 0.25 ml and 0.40 ml.

### Preparation:

Clean the treatment surface thoroughly to remove all calculus. In order to attain an optimal result, excess liquid/saliva should also be removed from the surface to be treated.

### Application:

Apply **VOCO Profluorid Varnish** evenly until the complete surface is covered with a thin film. Moisten the application area (either with a gentle rinse or by utilising the natural production of saliva) in order to ensure the **VOCO Profluorid Varnish** sets.

### Warnings, precautionary measures:

- Patients must be instructed to avoid eating solid food, drinking alcohol, cleaning their teeth and flossing for the first 4 hours after the treatment.
- Colophony can affect the polymerisation and bonding adhesion of composites.
- When using the cartridge, always use the intended, blunt application tips (length 32 mm, diameter 0.9 mm). **VOCO Profluorid Varnish** in the cartridge can be used multiple times. After application, remove the cannula and store the cartridge without the cannula (the septum rubber seals the cartridge tight). Attach a new cannula prior to the next application.
- **VOCO Profluorid Varnish** has an adhesive consistency corresponding to its intended purpose. When the tube is used, material residues on the thread may cause the cap to stick. Remove any excess material with a wipe moistened with ethanol if and as necessary.
- Oedematous swelling has been reported in rare cases, particularly after use on larger surfaces.
- Breathing difficulties have been reported in children with asthma in isolated cases where comparable products have been used.
- Patients with sensitive digestive systems may experience nausea after extensive treatment.
- In the case of hypersensitivity: remove the varnish with a soft toothbrush and dental floss before rinsing with warm water.
- No other products containing fluoride should be applied on the same day as the treatment.
- If the patient is taking fluoride tablets, they should skip their daily dose for a few days after treatment with **VOCO Profluorid Varnish**.
- Do not swallow the varnish.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

### Storage:

Storage at 4°C to 23°C. Do not use after the expiry date.

### Disposal:

Dispose of the product according to local regulations.

### Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's condition and a serious risk to public health that arise or could have arisen in association with the use of **VOCO Profluorid Varnish** must be reported to VOCO GmbH and the responsible authority.

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VC 60 AA1267 E1 0622 99 © by VOCO