**Product description:**

**Individo Lux blue opaque** and **Individo Lux blue transparent** are opaque and transparent light-curing resins respectively available in the form of contoured plates for maxillae and mandibles. They are ready for use and easy to handle without any odour nuisance.

**Individo Lux** remains workable for a long period of time when exposed to artificial light and daylight, curing quickly under halogen or UV light. The individual trays can be used directly after being made.

**Individo Lux** has a pleasant peppermint aroma.

**Indications:**

Production of individual impression and function trays for use in crown, bridge and total prosthetic techniques, for the production of bases for bite templates and for the production of occlusal registrations.

**Contraindications:**

**Individo Lux** contains peppermint oil, methacrylates and BHT. **Individo Lux** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

**Patient target group:**

**Individo Lux** is suitable for use in all patients without any age or gender restrictions.

**Performance features:**

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

**User:**

**Individo Lux** should only be used by a professionally trained dental practitioner.

**Use:**

Allow **Individo Lux** to reach room temperature in good time before use. Mark the margins for the individual tray on the plaster model. If necessary, use spacers. If no spacers are required, the plaster model should be isolated.

Remove the protective film and then place the plate on the model, mould it and trim the margins as indicated. The transparency of **Individo Lux blue transparent** allows very precise moulding. The excess material can easily be used to shape the handle.

**Light-curing:**

**Individo Lux** is polymerised using a standard light unit (halogen or UV light). Depending on the thickness of the contoured tray, light-cure the reverse for 3 to 5 minutes. A longer polymerisation time will improve the physical properties of the material. As such, the curing time should be prolonged if necessary.

**Finishing:**

The tray produced in this way is ready for use immediately or can be further processed without delay. Oxygen in the air can cause a slight inhibition layer, which can be wiped away using a cloth moistened with alcohol.

**Warnings, precautionary measures:**

- Exposure that is too short or not sufficiently intense will lead to a loss in material strength.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for the intended purpose.

**Storage:**

Store **Individo Lux** protected from light at 4°C to 23°C. Do not use after the expiry date.

**Disposal:**

Dispose of the product in accordance with local regulations.

**Reporting obligation:**

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Individo Lux** must be reported to VOCO GmbH and the responsible authority.

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VC 60 AA2400 E1 0822 99 © by VOCO