Carefully read instructions prior to use

**Product description:**

**Profibase** is a light-curing resin in pre-contoured plates for maxillae and mandibles for the fabrication of individual impression and functional trays for use in crown, bridge and total prosthetic technique to be used for the fabrication of bases of bite patterns as well as for dimensionally stable base plates for wax contouring in the total prosthetic technique.

The material is ready for use and easy to handle.

**Profibase** can be processed under room and day light for a long time and cures quickly under halogen and UVA light. Even after curing, **Profibase** maintains its pleasant peppermint flavour.

**Indications:**

**Profibase** is used for the fabrication of individual impression and functional trays for use in crown, bridge and total prosthetic technique.

**Contraindications:**

**Profibase** contains in an uncured condition BIS-GMA, diurethane-methacrylate, TEGDMA, BHT, peppermint oil. Do not use in cases of allergies or sensitivities.

**Method of use:****1. Fabrication of an individual tray**

Allow **Profibase** to reach room temperature before use. Mark margins for the individual tray on the plaster model. If necessary, use a space maintainer. If space maintainer is not necessary, insulate plaster model. Remove protective foil, mold plate onto the model and cut the desired rim as marked. A handle can easily be shaped from the excess material.

**Profibase** is then polymerized on the model in a standard light unit (halogen or UV light). Depending on the thickness of the tray, polymerize the reverse side for 3 - 5 min. A longer polymerization time improves the physical properties of the material. Therefore, prolong time of polymerization if necessary.

The individual tray is immediately ready for use and can be worked upon without delay. A slight inhibition layer, eventually caused by oxygen, can be easily removed with a tissue moistened with alcohol.

**2. Fabrication of a base plate**

Mark the margin of the planned prosthesis base on the plaster model. Block out undercuts as usual. Isolate the model thinly with an alginate isolation.

Carefully place a plate of **Profibase** onto the model, adjust the rims with a penknife and polymerize the model with base plate for 5 min. in a standard halogen or UVA light unit. One single exposure is sufficient.

**Supplementary information:**

For finishing the rims and the ridge, hard metal burs are most suited. Create cross-shaped retentions on the ridge before applying wax. Then a thin layer of sticky wax can be applied.

**Precautions:**

Insure sufficient and complete polymerization. A too short exposure time or low intensity of light may lead to a loss of stability.

**Storage:**

Store at temperatures between 39 °F - 77 °F (4 °C - 25 °C). Avoid direct sunlight. Excess material can be used again if stored under light-protection.

**Product order numbers:**

Combitkit of 25 maxillae and 25 mandibles	REF 2111
Pack 50 maxillae	REF 2112
Pack 50 mandibles	REF 2113

This material has been developed solely for use in dentistry.

Processing should be done strictly according to the instructions for use.

VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, either or consequential, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Descriptions and data constitute no warranty of attributes and are not binding.

**CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.**

No person is authorized to provide any information which deviates from the information provided in this instructions for use.

For questions or comments please call 1-888-658-2584 or 604-274-1975 or 604-377-9569.

**Keep the material out of reach for children.**

**For dental use only.**

An explanation of the symbols used in labeling can be found at [www.voco.dental/symbols](http://www.voco.dental/symbols)

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