

THE ALTERNATIVE
TO SURGERY

A M A R I S[®] Gingiva

HIGHLY ESTHETIC LIGHT-CURED COMPOSITE SYSTEM IN GINGIVA SHADES

Amaris® Gingiva

THE FUNCTIONAL AND ESTHETIC COMPOSITE IN GINGIVA COLORS FOR CHAIRSIDE USE

Dentists know of these clinical situations, ones which can be functionally restored with tooth-shaded composites, but not resolved esthetically.

Cervical defects caused by

- Decay
- Functional overload
- Periodontitis
- Abrasive brushing
- Malposition of teeth
- Hypomineralized hard dental tissue
- Age-related retreating gingival line

so far can be treated functionally, but not esthetically.

Defects in hard dental tissue could be restored with toothshaded composites. However, when the gingival retreats, this prolongs the optical appearance of the tooth beyond the gingival line.

If the restoration is carried out with tooth-shaded composites, both dentist and patient are rarely satisfied.


A disruption of the gingival “garland” line cannot be compensated with a tooth-shaded restorative.

Amaris® Gingiva is a composite restorative system specifically developed for the cervical area. One gingiva color and three opaquers, which can be mixed with each other, let you reproduce a multitude of gingival colors. Amaris Gingiva allows you to restore cervical defects both functionally and esthetically.

In laboratory-made restorative pieces, gingiva-colored materials have long been used successfully. Amaris® Gingiva now offers a chairside solution for this specific area of tooth anatomy.

Primary splinting of teeth with extended periodontal defects is a frequent application. Amaris® Gingiva allows to carry out this task without impairing the esthetics of the patient’s smile.

The gingiva-colored composite also allows you to restore exposed crown rims on otherwise intact PFM crowns, and thus satisfying the demands of your patients.

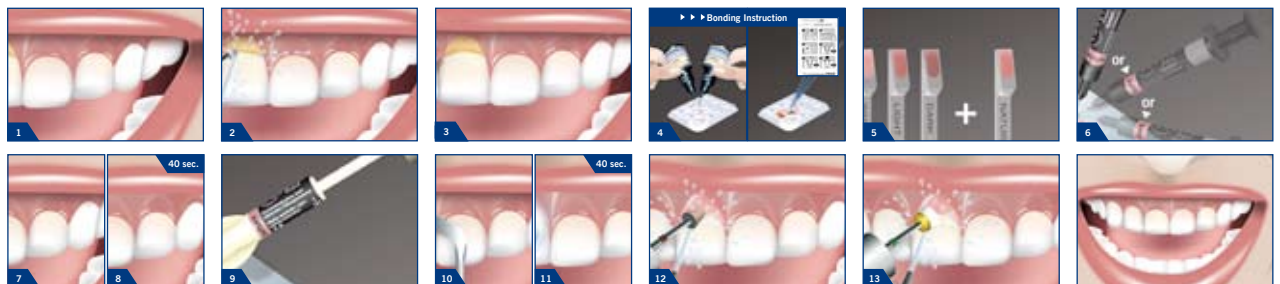

Amaris® Gingiva

CLINICAL APPLICATION



Initial situation on crowned teeth 9-11

Prof. Dr. H. Günay, Klinik für Zahnerhaltung, Parodontologie und Präventive Zahnheilkunde, Medizinische Hochschule Hannover, Germany



Cervical crown restorations with Amaris Gingiva



Initial situation on tooth 6

PD Dr. Dr. Bernd W. Sigusch, Poliklinik für Konservierende Zahnheilkunde des Universitätsklinikums Jena, Germany



Amaris Gingiva restoration after 4 days



Initial situation on teeth 6 and 7

Dr. Jan-Erik Schulz-Walz, Hamburg, Germany



Cervical restorations with Amaris Gingiva

Amaris® Gingiva

THE FUNCTIONAL AND ESTHETIC CHAIRSIDE COMPOSITE IN GINGIVA COLORS

Indications

- Composite restoration in gingiva colors
- V-shaped defects
- Exposed cervical areas
- Esthetic corrections also in the gingival area
- Primary splinting

Advantages

- Highly esthetic chairside composite restorative which can be adapted to a large variety of gingiva colors
- One basic color and three mixable opaquers can reproduce virtually all gingiva colors
- Cervical areas exposed after gingival recession and cervical V-shaped defects can be restored esthetically and functionally
- Gingiva colors now also for chairside use
- Advanced composite technique: Amaris® Know-How in gingiva colors for highly esthetic restorations beyond the gingival margin



Order information

REF 1970 Amaris Gingiva Set, 3 x 1.2 g Opaquers in the colors white, light and dark, 1 x 4 g Basic color "nature", accessories

REF 1971 Amaris Gingiva 1 x 4 g color "nature"

With compliments of your representative:

VC 8400 1970 US 1111 T

Call 1-888-658-2584