GIC Mixer

USER MANUAL

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MANUALE D'USO

IMPORTANT INFORMATION BEFORE FIRST USE

English

Deutsch

Français

Español

Italiano



1. Foreword

Thank you for purchasing this VOCO GIC Mixer capsule mixing device. Please read this user manual thoroughly and follow the instructions when operating the device. In addition, please keep this user manual in a safe place close to the device for future reference.

Intended use

This device mixes the powder and liquid in application capsules intended for use in dental treatments.


2. Product overview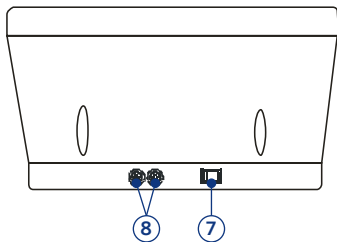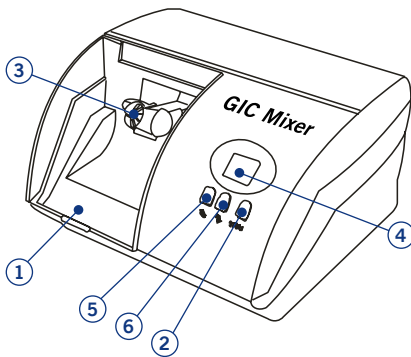

- ① Protective cover
- ② START button
- ③ Capsule holder
- ④ Time display
- ⑤ Increase time (seconds)
- ⑥ Reduce time (seconds)
- ⑦ On / Off switch
- ⑧ Fuse



3. Before first use

The device is fitted with four locking screws for safe transport. Please remove these locking screws before positioning the capsule mixer at its intended place of use and connecting it to the power supply.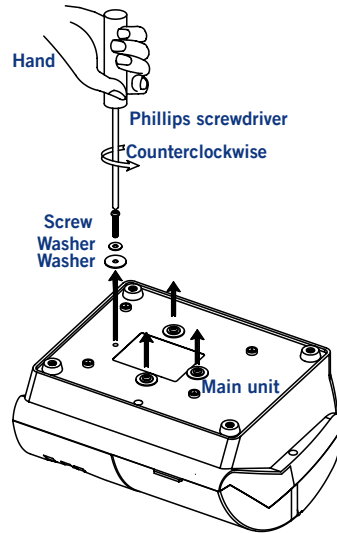

4. Operation

- 1) Select a suitable capsule product.
- 2) Activate the capsule in accordance with the manufacturer's specifications.
- 3) Open the protective cover on the device and insert the capsule in the capsule holder (see illustrations). Then close the protective cover again.

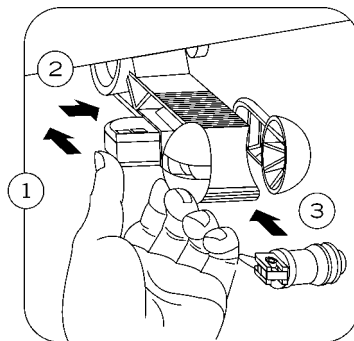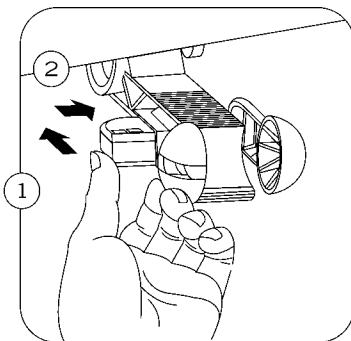- 4) Setting the time – Press the Up arrow button to increase the mixing time (in seconds) and the Down arrow button to reduce the mixing time (in seconds).
- 5) Press the START button to begin the mixing process. The device stops automatically once the set mixing time has elapsed. To interrupt the mixing process, press the START button again.
- 6) Once mixing is complete, open the protective cover and remove the mixed capsule, which can be used immediately.

5. Safety precautions and maintenance

- 1) Safety precautions
 - a) The four locking screws must be removed before the device is used for the first time.
 - b) Position the device on a flat surface with sufficient ventilation.
 - c) Ensure that the On/Off switch can be accessed easily and that the cable is connected securely to the device.
 - d) Close the protective cover before starting the device using the START button. Allow an adequate interval between multiple mixing processes so as to protect the device against overheating.
 - e) Please observe the instructions for use before using the device so as to avoid damage to the device and personal injuries.
- 2) Maintenance
 - a) Keep the device clean. The housing and capsule holder can be cleaned with a cloth moistened with alcohol.
 - b) The device should not be left unused for extended periods of time. When not in use, please disconnect the device from the power and store it in its original packaging.

6. Troubleshooting and repair

1) Troubleshooting


	Problem	Solution
1	The seconds are not shown on the LCD display	Change the fuse*
2	The device emits unusually loud noises during operation	Check that the locking screws have been removed from the device. If not, remove them.

*Note: Ensure that the device is off and the cable has been disconnected from the power supply before performing the steps listed above.

- 2) This device has been tested in compliance with the standard EN 60601-1-2:2001 Electromagnetic Compatibility. However, it is not possible to rule out minor electrical interference or that other devices can cause electromagnetic disturbances. Should disturbances occur, the following measures are recommended for resolving the problem:
 - a) Use different power sockets for the devices.
 - b) Increase the distance between the capsule mixing device and the susceptible unit.
 - c) Switch the device off and on again.
- 3) If it is not possible to identify the problem, please contact your local dealer or VOCO GmbH, Cuxhaven to find an optimal solution.
- 4) Repair
 - a) Our warranty period is one year from the date of purchase. If a defect occurs within the warranty period despite the device being used properly in accordance with the user manual, you are entitled to have the device repaired free of charge (does not apply to fuses or other wearing parts).
 - b) Please only use original spare parts from the manufacturer so as to guarantee their quality.

- c)  This symbol indicates the requirement for special disposal of electrical and electronic devices in EU countries. Do not dispose of the device as household waste. Contact your local recycling and waste disposal centre or your dealer for further information on the correct method of disposal. Please dispose of the device properly.

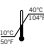7. Explanation of symbols

	Please observe the instructions for use		Conformity with applicable European directives
	Recommended storage and operating temperature		Fragile, handle with care
	Protect against moisture		Do not dispose of as household waste

You can find explanations of the symbols used at www.voco.com/us/symbols

8. Technical data

Input voltage	EU: ~ 220 / 230 / 240 V, 50 Hz US: 110 V, 60 Hz
Power rating	400 VA
Oscillations per minute	ca. 4600
Noise level	≤ 65 dB
Selectable mixing time	1 sec. - 99 sec.
Intermittent operation	1 minute interval for every 3 minutes of operation
Fuse	EU: F2AL250V US: F3AL250V
Protection class	Class II

Housing	IPX0
Operating conditions	Temperature: 5 – 40 °C Rel. humidity: ≤ 80 % Ambient air pressure: 850 – 1060 hPa
Storage conditions	Temperature: - 40 – 50 °C Rel. humidity: ≤ 80 % Ambient air pressure: 500 – 1060 hPa

9. Warranty

VOCO gives a 12-month warranty period if the device cannot be used as intended due to quality defects. This warranty begins on the day of purchase. VOCO reserves the right to either repair the device or replace individual parts or the entire device at its discretion. This warranty does not cover wearing parts or fuses.

10. General information

- We cannot accept any liability for malfunctions or damage resulting from improper use, the removal of components, modifications or inadequate maintenance.
- We cannot accept any liability for malfunctions or damage resulting from operation in unsuitable environments.
- We cannot accept any liability for malfunctions or damage resulting from acts of God.
- The warranty shall also be void in the case of product defects which can be definitively attributed to natural disasters or excessive temperatures or humidity levels that do not conform to the specifications in this user manual.

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