# VOCO

## Admira Protect

USA | EN Instructions for use

Carefully read instructions prior to use

### Product description:

**Admira Protect** is a light-curing, one-component Ormocer®-based preparation for the treatment of dentin hypersensitivity. In most cases, hypersensitivity is reduced after only one application. Moreover, **Admira Protect** safeguards the treated teeth against external influences and abrasion.

### Indications:

- treatment of hypersensitive dentin
- treatment of cervical areas (including crown margins, after tooth cleaning or tartar removal, exposed cervical areas etc.)

### Contraindications:

**Admira Protect** contains Bis-GMA, HEMA, BHT, acetone, ormocer and organic acids. Do not apply in cases of known allergies to these ingredients.

### Method of use:

#### 1. Preparation

Clean teeth to be treated with fluoride-free cleaning paste or pumice on a rubber cup or rotating brush.

Isolate areas to be treated with cotton rolls. Remove excess moisture with an oil-free airjet. Do not overdry dentin. The dentin surface should be slightly moist.

After cleaning, avoid any contamination with blood or saliva.

#### 2. Dispensing

##### When using the bottle:

Make sure to have the vent-hole over the dispensing dropper when tilting the bottle. Insert a **Single Tim** into the dropper and let soak, or dispense **Admira Protect** onto a mixing well.

##### When using the *SingleDose*:

Remove a *SingleDose* on perforation and turn the printed side upwards. Break through the foil with a **Single Tim**. Enlarge the gap and moisten the brush with liquid by circle movements.

#### 3. Application

Apply **Admira Protect** evenly with **Single Tim** or a disposable brush on all dentin surfaces to be treated and let act for 20 sec. Then disperse **Admira Protect** with a faint airjet and light-cure with a conventional polymerization device for 10 sec. The light output should be a minimum of 500 mW/cm<sup>2</sup> on halogen polymerization devices and 300 mW/cm<sup>2</sup> on LED devices.

Apply a second layer of **Admira Protect**, disperse it with a faint airjet and light-cure for an additional 10 sec.

#### 4. Removal of the inhibition layer

Remove the oxygen-inhibited layer (soft surface) with a cotton pellet or a cotton roll. Check the gingival margins and remove any residual material, if present.

### Precautions:

Phenolic substances (e.g. eugenol-containing temporary materials) disturb polymerization and may not be brought in contact with **Admira Protect**.

Avoid contact to the mucosa. Upon contact with the eyes, flush immediately with plenty of water and consult a physician.

### Storage and application:

Tightly close **Admira Protect** bottles immediately after use. Store at temperatures between 39°F - 73°F / 4°C - 23°C (storage at 40°F - 45°F / 5°C - 8°C prolongs the shelf-life of the preparation). Do not expose **Admira Protect** to direct sunlight or the chairlight.

### Ordering Information:

<b>Admira Protect</b>	
4,5 ml bottle, accessories	REF 1650
<b>Admira Protect <i>SingleDose</i></b>	
Set 50 <i>SingleDose</i> , accessories	REF 1651
Pack 200 <i>SingleDose</i> , accessories	REF 1652

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use.

VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Descriptions and data constitute no warranty of attributes and are not binding.

**CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.**

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

**Keep this material out of reach of children.**

**For dental use only.**

An explanation of the symbols used in labeling can be found at [www.voco.dental/symbols](http://www.voco.dental/symbols)

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VOCO GmbH