VOCO

Bifix® Temp

USA | EN

Instructions for use

Carefully read instructions prior to use

Product description:

Bifix Temp is a dual-curing, composite-based temporary luting material. The balanced translucency of **Bifix Temp** enables highly aesthetic results to be achieved for non-permanent luting of temporary restorations made from crown and bridge materials as well as temporary CAD/CAM-fabricated pieces. The low film thickness ensures precise placement of temporary restorations at all times. The stability of **Bifix Temp** facilitates particularly simple and reliable removal of excess material.

Working time: > 1.5 min (73 °F/23 °C)
Setting time: < 4 min (99 °F/37 °C)

Indications:

- Non-permanent luting of temporary restorations.

Contraindications:

Bifix Temp contains dimethacrylates and diacrylates, benzoyl peroxide, amines and BHT. **Bifix Temp** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.

Caution:

The surface of composite build-ups and restorations must be coated with a thin layer of insulation (glycerin gel, Vaseline) in order to prevent **Bifix Temp** from luting permanently.

The use of (meth)acrylate-based bonding agents and protective varnishes prior to luting with **Bifix Temp** must be avoided as this can result in a permanent bond.

Application:

1. Preparation:

Ensure that the surface is clean and dry; if necessary it should be cleaned and dried. Follow the instructions for application on composite build-ups and restorations (see **caution** above).

2. Placing the restoration:

Remove the protective cap from the QuickMix syringe. Due to technical reasons, there may be slight differences in the filling levels of the syringe chambers prior to first use. Therefore remove the syringe cap and extract material until both chambers deliver uniformly. Then fit a mixing tip type 9. The notches on the syringe and the mixing tip must line up exactly. Turn the mixing tip 90° clockwise to lock the tip into place. Apply **Bifix Temp** to the dry temporary restoration. Then place the restoration by applying gentle pressure, generating excess material in the process. The working time at room temperature is at least 1.5 min. Leave the used mixing tip on the syringe to seal it until the next application.

3. Removal of excess material:

Any excess material can be removed using the following methods:

- Wait until the material turns elastic (monitor change in consistency as a consequence of intraoral chemical curing) and remove excess material using a suitable instrument.
- Alternatively, the surplus material can be rendered elastic after short curing with a polymerization light (around 2 s) and then removed using a suitable instrument.
- If not cured, **Bifix Temp** can be removed using a foam pellet or a similar object.

Removal of excess material following brief curing is recommended. The restoration must be held in position during the polymerization phase. Contact points and interproximal areas must be cleaned with dental floss. There must be no excess material remaining in the sulcus as this can cause gingival irritation.

4. Curing:

Self-curing is completed 4 min after insertion. The polymerization time can be controlled by additional light-curing. For this, each side is cured for a minimum of 10 s using a polymerization light with an intensity of at least 500 mW/cm².

Notes and precautions:

Bifix Temp is eugenol-free and well suited for use prior to subsequent adhesive permanent luting as it has no influence on the curing of the materials used. Only apply **Bifix Temp** using the designated mixing tip, manual mixing is not advisable due to oxygen inhibition. If, during the course of treatment, the temporary restoration is to be luted again, the application of a thin layer of, for example, glycerin gel or Vaseline to the restoration prior to luting can make it easier to subsequently remove any **Bifix Temp** residue from the temporary restoration. In case of contact with the eyes, immediately rinse with plenty of water and seek a medical advice.

When using the cartridge/dispenser make sure to avoid cross-contamination between patients. Employ protective sleeves or use hospital grade disinfectants which do not contain phenols, oxidizing agents or amines. Do not immerse the cartridge or QuickMix syringe in disinfectant solutions.

The warranted product characteristics can only be achieved when using the supplied original mixing tips, which are also available as accessories.

Storage:

Store at temperatures of 39 °F to 73 °F (4 °C to 23 °C).

Do not use after expiry date.

Refrigeration is recommended.

Allow **Bifix Temp** to reach room temperature before use!

Product Ordering Information:

QuickMix syringe 5 ml universal, accessories
Mixing tips type 9, 50 pcs.

REF 1055
REF 2188

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use.

VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

Keep this material out of reach of children.

For dental use only.

An explanation of the symbols used in labeling can be found at www.voco.dental/symbols

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VOCO GmbH