Futurabond DC

Carefully read instructions prior to use

Product description:

Futurabond DC is a dual-curing, self-etching bond reinforced with nano-particles for creating a durable bond between tooth substance and light-, self- and dual-curing restoratives without marginal leakage. This all-in-one adhesive consists of 2 components, which eliminates the need for a separate etching step, yet still achieves the same adhesive properties as total-etch bonds.

Futurabond DC tolerates residual moisture and can be used universally with all light-, self- and dual-curing composites, compomers and ORMOCERS®.

Indications:

- Direct self- or dual-curing composite restorations and core build-ups
- Direct light-curing composite/compomer/ORMOCER®-based restorations
- Fixing root pins/posts with dual- or self-curing composite cements
- Indirect restorations; with the application of dual-curing/self-curing composite cements for bonding inlays, onlays, crowns and bridges

Contraindications:

Futurabond DC contains organic acids, BIS-GMA, HEMA, TMPTMA, campherchinon, amines (DABE), BHT, catalysts, fluorides and ethanol. Do not use in patients with known allergies to these ingredients.

Warnings and precautionary measures:

- Do not fold blister before activation. Blisters with obvious damage such as creases or kinks should not be used.
- Do not damage either the blister foil or tab on either the top or the bottom side with fingernails.
- Do not use any type of instrument for activation. Only the thumb and index finger should be used for activation with the thumb placed exactly on the mark and the index finger placed exactly on the opposite side of the package.
- Activation and subsequent puncturing of the foil with a **Single Tim** should not take place in close proximity to the eyes.
- Never simultaneously press the reservoir from both sides or press the liquid back and forth between the reservoirs.
- Phenolic substances (e.g. temporary materials containing eugenol) disturb polymerization and therefore should not be used with **Futurabond DC**.
- Avoid contact with the oral mucosa. Contact of the oral mucosa with **Futurabond DC** may result in a whitish discoloration caused by the coagulation of proteins. This is a reversible occurrence that subsides within a few days. In case of contact with the eyes, rinse with plenty of clean water and consult a physician. **Single Tim** application brushes provide precise and thin application of **Futurabond DC**.

Application:

- 1. Preventing contamination**
A dry working field is essential due to the subsequent application of the restorative. Avoid contamination with blood or saliva. Using a rubber dam is recommended.
- 2. Preparation**
Prepare the cavity according to the principles of the adhesive restoration technique. Rinse cavity thoroughly with water. Remove excess moisture with a faint air jet. Do not overdry dentin. The dentinal surface of the cavity should remain slightly moist. If the tooth has not been prepared (e.g. cervical defect) the surface must be thoroughly cleaned and enamel margin roughened with a finishing diamond. Areas in close proximity to the pulp should be protected with a suitable calcium hydroxide-based liner.
- 3. Mixing**
Futurabond DC is dual-curing and therefore must be applied immediately after mixing.
When using the bottled version:
Mix 1 drop of Liquid 1 and 1 drop of Liquid 2 on a mixing palette with a **Single Tim** for approximately 2 sec (produces a self-etching adhesive).
When using SingleDose:
Detach a *SingleDose* by folding along the perforation and turn the printed side upwards. To activate, hold the *SingleDose* blister between thumb and index finger and press on the blister where indicated "press here" in blue. This injects Liquid 1 into Liquid 2 in the dispensing bay in a 1:1 ratio. Pierce the foil of the dispensing bay with the **Single Tim**, enlarge the hole with a circular movement and stir well to create a homogeneous mixture.
- 4. Futurabond DC application**
Apply the adhesive in a layer of medium thickness to the enamel/dentin and rub into the tooth surface for 20 sec.
Caution: The mixed bonding is lightcuring and thus an intensive exposure to ambient light should be avoided. The OP-Light during the application should be dimmed.
- 5. Drying**
Dry the adhesive layer for at least 5 sec with an air syringe.

6. Polymerization

6.1. Direct restoration

Light-, self- and dual-curing materials

The bonding layer must be polymerized with blue light (halogen or LED lights) for 10 sec when using light-, self- and dual-curing restoratives.

6.2. Indirect restoration

Light-curing luting composites

The bonding layer must be polymerized with blue light (halogen or LED lights) for 10 sec when using light-curing luting materials.

Self- or dual-curing luting materials

When using dual- or self-curing luting materials, **Futurabond DC** also provides excellent adhesion without light-curing. **Futurabond DC** can also be light-cured for 10 sec with bluelight (halogen or LED lights) as an optional step.

Luting root posts

- a. Massage the mixed **Futurabond DC** (liquid 1 and 2) into the occlusal surface of the core and root canal for 20 sec with an applicator. Disperse solvent with an oil-free air-jet for 5 sec. Remove excess material by dabbing it off with an absorbent strip of paper. Do **not** light-cure the bonding layer.
- b. Having prepared the root post in accordance with the manufacturer's instructions, insert it into the root canal using a suitable luting material such as **Rebilda DC** or **Bifix QM** (see respective instructions for use). The aim is to obtain a slight excess of cement.
- c. Light-cure for at least 40 sec to fix the root post and cure the **Futurabond DC** applied to the occlusal surface.

7. Advice on luting Maryland Bridges

For luting Maryland bridges, the enamel must either be roughened or conditioned with a phosphoric acid etching gel before applying **Futurabond DC**.

Self-etching bonding materials are less effective on unprepared enamel. Overapplication of bonding material to enamel that has not been ground or bevelled can lead to margin discoloration. For optimum adhesion results with **Futurabond DC**, the enamel should be prepared in advance in class III, IV, V cavities and with diastema closures.

Advice for storage and usage:

Tightly close **Futurabond DC** bottles after use. Store in an upright position to guarantee flow-back of the liquid. Store at temperatures between 39°F - 73°F (4°C - 23°C). Allow **Futurabond DC** to reach room temperature before use. Do not expose **Futurabond DC** to direct sunlight or the chairlight. Do not use **Futurabond DC** after the expiration.

PRODUCT ORDERING INFORMATION:

Futurabond DC (bottle)	
Set each 4ml liquid 1/2, accessories	REF 1163
Futurabond DC SingleDose	
- Kit 50 <i>SingleDose</i> , accessories	REF 1164
- Pack 200 <i>SingleDose</i> , accessories	REF 1165
100 Single Tim	REF 2247

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith. Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

Keep this material out of reach of children. For dental use only.

US Patent No. 7,556,147 B2
US Patent No. 7,677,417 B2

An explanation of the symbols used in labeling can be found at www.voco.dental/symbols

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VOCO GmbH