VOCO

Grandio Core Dual Cure

USA | EN

Instructions for use
In accordance with DIN EN ISO 4049

Carefully read instructions prior to use

Product description:

Grandio Core Dual Cure is a dual-curing, radiopaque, flowable composite for core build-up, with very good mechanical properties.

Grandio Core Dual Cure contains 77% w/w inorganic fillers in a methacrylate matrix.

Shades:

Grandio Core Dual Cure is available in three different shades:

- dentin shade for high aesthetic demands, e.g. under all-ceramic restorations or highly translucent composite restorations
- blue to indicate the transition between the material and the dental tissue, as a base for metal crowns or faced crowns with a metal base
- white as an option for indicating preparation borders, while simultaneously maintaining aesthetic requirements

Indications:

- Core build-up of vital and non-vital teeth
- Luting of root posts

Contraindications:

Grandio Core Dual Cure contains methacrylates, amines, benzoylperoxide and BHT. **Grandio Core Dual Cure** should not be used in cases of known hypersensitivities (allergies) to any of these ingredients.

Method of application:

Moisture control:

Ensure that the working field is sufficiently dry. The use of a rubber dam is recommended.

Cavity preparation:

In principle, the cavities should be prepared according to the rules for adhesive restorative treatment, i.e. preparation should be minimally invasive in order to protect healthy dental hard tissue. Then clean and dry the cavity.

Lining the cavity:

Take appropriate measures in areas close to the pulp.

Bonding materials:

Grandio Core Dual Cure is applied with a dentin/enamel bond indicated for dual-curing composites, using the adhesive technique. Please follow the respective instructions for use in regard to preparation and application.

Preparing the QuickMix syringe:

For technical reasons, there may be slight differences in the filling levels of the two syringe chambers prior to initial use. Therefore remove the syringe cap and continue to extrude material until it flows evenly from both outlets. Then attach a type 11 mixing tip. The notches on the syringe and the mixing tip must be lined up. The mixing tip is fixed by rotating clockwise through 90°. If necessary, an additional intraoral tip type 4 can be placed on the end of the mixing tip. The material is automatically mixed in the correct proportions inside the mixing tip as it is dispensed and can be applied immediately. All QuickMix syringes are equipped with a removable plunger. The plunger can easily be pulled backwards out of the syringe and then reinserted. The seals in the syringe do not change their position whilst this is being done. This prevents contamination of the materials in the syringe.

Application of core build-up materials:

Place matrix around the prepared tooth or apply shaping aid and **Grandio Core Dual Cure** directly from the QuickMix syringe. **Grandio Core Dual Cure** has a working time of approx. 1.5 min (73°F/23°C).

Light-curing:

Apply mixed material into the cavity in increments of no more than 2 mm and polymerize for 40 s each time using an LED/halogen lamp with a minimum light output of 500 mW/cm². Hold the light emitting window of the polymerization lamp as close to the surface of the core build-up as possible, as curing may otherwise be insufficient. The restoration can be finished immediately after light-curing.

Chemical curing:

If light-curing is not carried out, the build-up can be processed 5 min after insertion into the mouth at the earliest.

Luting of root posts:

Use an adhesive indicated for dual-curing composites for the luting of root posts. Please follow the respective instructions for use in regard to preparation and application.

1. Using the QuickMix syringe:

It is recommended that **Grandio Core Dual Cure** is applied with the fine intra-oral tip type 1 directly into the root canal. Do not use Lentulos! In addition, **Grandio Core Dual Cure** can be applied to wet the root post, when the latter has been prepared according to the manufacturer's instructions. Insert the post making sure to leave a slight excess of cement. Due to the increased temperature and the contact with the bonding agent, the working time of **Grandio Core Dual Cure** in the root canal is reduced to 40 s.

2. To fix the root post and to cure the occlusally applied bonding, the material should be light-cured for at least 40 s.

3. For core build-up, proceed as described in "Application of core build-up materials".

Information / precautionary measures:

- No known side effects. Sensitization in hypersensitive patients, however, cannot be excluded.
- Phenolic substances, especially preparations containing eugenol and thymol, lead to curing failures in composites. The use of zinc oxide eugenol cements or other eugenol-containing materials in combination with composites is therefore to be avoided.
- The use of one-bottle adhesive systems is to be avoided, as an optimal bond with chemically or dual-cured materials cannot, as a general rule, be guaranteed.
- Isolate the core build-up for taking impressions or fabricating temporary crowns/bridges (it may be sufficient to moisten it thoroughly), in order to prevent any bonding with these materials.
- Avoid contact with the oral mucosa.
- In cases of purely chemical curing, low temperatures (e.g., when working on the anterior region) can lead to an increase in setting time.
- The warranted product characteristics can only be achieved when using the supplied original mixing tips, which are also available as accessories.

Storage and application instructions:

After use, leave the mixing tip on the QuickMix syringe or replace the cap. Do not expose **Grandio Core Dual Cure** to direct sunlight or the operating light. Store at temperatures of 39 °F - 73 °F (4 °C - 23 °C). Refrigeration is recommended. Bring to room temperature before use. Do not use after the expiry date.

Product ordering information:

QuickMix syringe 10 g dentin	REF 1910
QuickMix syringe 10 g blue	REF 1911
QuickMix syringe 10 g white	REF 1912

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

**Keep this material out of reach of children.
For dental use only.**

An explanation of the symbols used in labeling can be found at www.voco.dental/symbols

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VOCO GmbH Phone +49 (4721) 719-0