# IonoStar Molar

In accordance with DIN EN ISO 9917-1  
Carefully read instructions prior to use

**Product description:**

**IonoStar Molar** is a radiopaque glass ionomer bulk filling material in user-friendly capsules for direct application. In addition to high compressive strength, it boasts good adhesion to enamel and dentin and releases fluoride ions.

**IonoStar Molar** is available in the shades A1, A2 and A3. The usable quantity of cement per capsule is at least 0.15 ml.

**Indications:**

- Restorations of non-occlusion bearing class I cavities
- Semi-permanent restorations of class I and II cavities
- Restorations of cervical lesions, class V cavities, root caries
- Restorations of class III cavities
- Restoration of deciduous teeth
- Base/liner
- Core build-up
- Temporary restorations

**Contraindications:**

**IonoStar Molar** contains polyacrylic acid. **IonoStar Molar** should not be used in case of known hypersensitivities (allergies) to this ingredient.

**Method of use:**

**Preparation:**

Prepare the cavity using a minimally invasive approach (removing only carious tooth substance). There is no need for undercuts. The layer thickness of the restorations must not be less than 1 mm. Avoid thinly tapering margins. Clean the cavity, rinse with water and dab or blow dry. Do not over dry the dentin: it should have a moist, shiny appearance. If the residual dentin thickness is insufficient, the area close to the pulp should be covered with a calcium hydroxide product.

**Activating the application capsule and mixing the material:**

Only extract the application capsule from the tubular bag immediately before use. During activation, the liquid from the plunger must be pressed into the inside of the capsule as fully as possible. To achieve this, the end of application capsule where the plunger of liquid protrudes is slowly pressed down onto a hard surface (e.g. a table) as far as the plunger will move (see fig. 1). The capsule must only be activated directly before use and loaded immediately into a high-frequency mixer with approx. 4,000 oscillations per minute. Mix the activated application capsule for 10 - 15 seconds (mixing for longer reduces the working and setting times), remove it from the mixer without delay and insert it in the holder of an application device (e.g. AC Applicator, see corresponding instructions for use).

**Application:**

Carefully open the outlet by lifting the cannula until you feel it stop (see fig. 2), and apply the material directly into the cavity within 15 seconds of the end of mixing by compressing the lever on the applicator slowly and evenly. The cannula can be aligned as required by turning the whole application capsule inside the holder. Working time and consistency may be influenced by the respective type of capsule mixer and mixing time.

Fig. 1: Activating the capsule

Fig. 2: Opening the capsule

Following mixing, **IonoStar Molar** has a working time of 1.5 minutes (at room temperature). Prevent the material from being contaminated with moisture and from drying out. If it is not possible to keep the working area dry, a protective varnish should be applied after the contours/modelling have been finished (see corresponding instructions for use). The restoration can be finished, e.g. with a diamond bur or a polisher, 5 - 6 minutes after the start of mixing at the earliest. A protective varnish should then be applied again (see corresponding instructions for use).

Mixing time (73° F (23 °C))	10 - 15 s
End of mixing to application (73° F (23 °C))	15 s
Working time (73° F (23 °C))	1.5 min
Setting time (99° F (37 °C))	3 - 4 min
Total setting time	5 - 6 min

**Information, precautionary measures:**

Do not apply directly to dentin near the pulp or to exposed pulp. In case of contact with the eyes, rinse immediately with plenty of water and consult a specialist. Rinse the instruments used immediately with cold water to avoid material curing. Ensure that the AC capsule is mixed immediately after activation.

**Storage instructions and application method:**

Store **IonoStar Molar** in a dry place at temperatures of 59 °F to 73 °F (15 °C to 23 °C). Only remove it from the foil immediately before use. Do not use after expiry date.

**Product Ordering Information:**

<b>IonoStar Molar</b> – Set Application Capsule 50 pcs. (10 × A1, 30 × A2, 10 × A3)	REF 2520
<b>IonoStar Molar</b> – Set Application Capsule 150 pcs. A2, Applicator type 1	REF 2522
<b>IonoStar Molar</b> – Set Application Capsule 150 pcs. A3, Applicator type 1	REF 2523
<b>IonoStar Molar</b> – Application Capsule 20 pcs. A1	REF 2524
<b>IonoStar Molar</b> – Application Capsule 20 pcs. A2	REF 2525
<b>IonoStar Molar</b> – Application Capsule 20 pcs. A3	REF 2526
<b>IonoStar Molar</b> – Application Capsule 150 pcs. A2	REF 2532
<b>IonoStar Molar</b> – Application Capsule 150 pcs. A3	REF 2533

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use.

VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Descriptions and data constitute no warranty of attributes and are not binding.

**CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.**

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

**Keep this material out of reach of children.  
For dental use only.**

An explanation of the symbols used in labeling can be found at [www.voco.com/us/symbols](http://www.voco.com/us/symbols)

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**VOCO GmbH** Phone +49 (4721) 719-0