Carefully read instructions prior to use

Provicol/Provicol C/Provicol QM, a eugenol-free cement with calcium hydroxide, is easy to process and does not influence the setting of resin-based filling and luting materials.

Provicol/Provicol C/Provicol QM is recommended for patients with known allergies to eugenol.

Provicol/Provicol C/Provicol QM contains calcium hydroxide to support tooth vitality.

Indications:

Provicol/Provicol C/Provicol QM is used for cementing temporary crowns and bridges, inlays and onlays.

Contraindications:

Provicol/Provicol C/Provicol QM contains zinc oxide, magnesium oxide, calcium hydroxide, fatty acids, modified rosin and vegetable oils. Do not use in cases of known sensitivities or allergies.

Method of use:

Provicol QM (in the QuickMix syringe)

Due to technical reasons slight differences in the filling level between the syringe chambers can occur prior to the first use. Therefore remove syringe cap and squeeze the material out until the material is delivered evenly from both chambers. Place and lock the mixing tip type 10 with a 90° turn clockwise. The material is automatically mixed in the mixing tip.

All QM syringes come with a detachable piston. The piston can simply be pulled from the QM syringe and replaced without causing any material leakage. The plugs in the syringe chambers remain in place, preventing cross-contamination of the materials.

Provicol C (material in cartridges)

Insert **Provicol C** cartridge into a compatible dispenser with a suitable piston. Due to technical reasons, slight differences in the filling level between the cartridge chambers can occur. In this case, remove cartridge cap and squeeze the material out until the material is delivered evenly from both chambers. Then, insert a mixing tip type 6 and lock with a 90° turn clockwise. By pressing the lever handle slowly and evenly, the material is extruded and, in the process, automatically mixed in the correct ratio. Do not remove the mixing tip from the cartridge after use. Wait to replace the used mixing tip until immediately before the next application. When changing the mixing tips, check the opening of the cartridge to confirm material can flow freely.

Provicol (material in tubes)

Provicol should be used at room temperature (approx. 64 - 73°F [18 - 23°C]). Lower temperatures (e.g. using a cooled glass plate) extend the working time. The **Provicol** catalyst and base paste are mixed in a 1:1 w/w proportion. Therefore dispense two equal lengths of base and catalyst paste on a mixing pad of strong, non-absorbing material or a glass plate. Then mix the two pastes for approximately 20 sec. until the material reaches a homogeneous consistency. Use a spatula made of plastic or stainless steel for mixing.

To obtain a softer material and to keep **Provicol** workable for a longer period of time, add a small amount of vaseline.

A harder and faster setting material is obtained by adding zinc oxide (temporary sealing).

Application:

Apply a thin layer of **Provicol/Provicol C/Provicol QM** inside the temporary crown or bridge and affix it to the prepared tooth.

Provicol remains workable for about 1.5 - 2 min, **Provicol C/Provicol QM** about 1 min. All materials set in the mouth within 4 - 6 min.

Place the temporary into the mouth. Remove any material excess after 3 - 5 min.

Precautionary measures:

Pulp protection (e.g. calcium hydroxide cement) is recommended in cases of exposed pulp or inadequate dentin thickness.

The sensitivity of cores can be reduced by using desensitizing preparations prior to temporary luting. Remove these materials completely before permanent luting (Do not use preparations containing eugenol!).

Extensive superficial and multi-surfaced cavities as well as temporary long-term sealing work should be treated with a harder cement.

The warranted product characteristics can only be achieved when using the supplied original mixing tips, which are also available as accessories.

Supplementary instructions/information:

Moisten cores with water when using Provicol in tubes.

Moisture (humid cores) further facilitates removal of the temporary.

Before removing **Provicol/Provicol C/Provicol QM** residues, spray with a cooling spray and wait for a short time.

Store **Provicol/Provicol C/Provicol QM** at temperatures between 39 - 73°F (4 - 23°C). Close tubes tightly after use.

Provicol/Provicol C/Provicol QM corresponds to EN 23107 and ISO 107/1988.

Ordering Information:

Provicol QM

5 ml QuickMix syringe, mixing tips type 10 REF 1074

Pack 50 mixing tips type 10 REF 2189

Provicol C

Pack 2 x 65 g in cartridges, mixing tips type 6 REF 1076

Pack 50 mixing tips type 6 REF 2185

Provicol

Set 25 g base, 25 g catalyst in tubes REF 1075

Bulk pack 5 x 25 g base, 5 x 25 g catalyst in tubes REF 1078

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith. Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call: 1-888-658-2584.

Keep this material out of reach of children.

For dental use only.

An explanation of the symbols used in labeling can be found at www.voco.dental/symbols

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