Provicol® QM Plus

in accordance with DIN EN ISO 3107

Carefully read instructions prior to use

Product description:

Provicol QM Plus is an easy-to-use, eugenol-free, temporary luting cement, which does not affect the setting of composite-based restoratives and luting materials. The calcium hydroxide contained in **Provicol QM Plus** helps maintain tooth vitality.

Working time: ≥ 1 min (73 °F (23 °C))
 Setting time: ≤ 4 min (99 °F (37 °C))

Indications:

- Temporary cementation of provisional restorations (crowns, bridges, inlays and onlays)
- Temporary cementation of definitive restorations (crowns, bridges, inlays and onlays)

Contraindications:

Provicol QM Plus contains modified colophony. **Provicol QM Plus** should not be used in case of known hypersensitivities (allergies) to this ingredient.

Method of use:

1. Preparation:

Ensure that surfaces are clean before cementation begins (see supplementary advice below). Remove the protective cap from the QuickMix syringe. Due to technical reasons, there may be slight differences in the filling levels of the syringe chambers prior to first use. Therefore remove the syringe cap and extract material until both chambers deliver uniformly. Then fit a mixing tip type 10. The notches on the syringe and the mixing tip must line up exactly. Turn the mixing tip 90° clockwise to lock the tip into place.

The warranted product characteristics can only be achieved when using the original Sulzer Mixpac mixing tips as approved by **VOCO**. All the mixing tips for use with our respective products are available from **VOCO**.

2. Placing the restoration/application:

Apply **Provicol QM Plus** to the dry temporary restoration and/or the prepared tooth surface. Then place the restoration by applying gentle pressure, generating excess material in the process. The working time at room temperature is at least 1 min. Leave the used mixing tip on the syringe to seal it until the next application. Alternatively, re-seal the syringe with the protective cap after application.

3. Curing:

Any excess material can be removed using the following methods:

- Remove excess amounts of not fully cured **Provicol QM Plus** using a foam pellet or similar.
- Wait for material to cure fully (observation of progress in the mouth) and then remove excess cured cement using a suitable instrument.

Contact points and interproximal areas need to be cleaned with dental floss. The sulcus must be free of material residues, as they may irritate the gingiva.

Curing is completed 4 minutes after insertion. The restoration must be held in position during the polymerization phase.

4. Removal:

Use a suitable instrument to remove the restoration and completely remove all **Provicol QM Plus** residues before final cementation. This can be done with a suitable instrument (curette, scaler, probe) and, if required, with polishing paste.

Warning/precautionary measures:

In case of contact with the eyes, immediately rinse with plenty of water and seek medical advice. The sensitivity of cores can be reduced by using desensitizing preparations prior to temporary cementation.

Appropriate measures for pulp protection are recommended in cases of exposed pulp or insufficient residual dentin thickness. A harder cement is the better choice for multi-surface cavities and those which have an extensive surface area, as well as long-term temporary obturations.

Provicol QM Plus residues are easily cleaned off restorations after removal if they are briefly treated with cooling spray outside the mouth.

Supplementary advice:

Wetting prepared teeth with a little water before cementation makes subsequent removal of the restoration easier.

Storage:

Store at temperatures of 39 °F to 73 °F (4 °C to 23 °C).

Do not use after expiration date.

Allow **Provicol QM Plus** to reach room temperature before use!

PRODUCT ORDERING INFORMATION:

QuickMix syringe 5 ml, accessories

REF 1098

Mixing tips type 10, 50 pcs.

REF 2189

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

**Keep this material out of reach of children.
 For dental use only.**

An explanation of the symbols used in labeling can be found at www.voco.dental/symbols

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VOCO GmbH