VOCO

Registrado X-tra

USA | EN Instructions for use

Carefully read instructions prior to use

Product description:

Registrado X-tra is an addition-curing silicone for bite registration with extreme final hardness which sets rapidly. Highly thixotropic properties prevent it dripping from the teeth and allow for application in thicker layers. The special temperature-controlled setting formula eliminates time pressure during application and has an extremely short setting time in the mouth. After only 40 s the registration can be removed without breaking or warping.

Registrado X-tra can be easily trimmed with a scalpel or rotating instrument and stays dimensionally stable.

Indications:

Registrado X-tra is suitable for occlusal bite registration, fixing of face bow registrations, and it is a location material for intraoral pin tracing templates.

Contraindications:

Registrado X-tra contains polydimethylsiloxanes and platinum catalyst. Do not use in case of allergies to these ingredients.

Do not use with condensation-curing silicones. Latex gloves, mouthrinse and retraction solutions can also have a negative influence on the setting process.

The warranted product characteristics can only be achieved when using the supplied original mixing tips, which are also available as accessories.

Application:

1. Preparation:

Use a **VOCO** Dispenser type 2 or any other suitable dispenser. Due to technical reasons there may be slight differences in the filling levels of the two cartridge chambers. Therefore, remove cartridge cap after placing the cartridge into the dispenser type 2 and extract material until it is delivered uniformly from both chambers. Residues from other materials (e. g. impression materials) or solutions may interfere with the setting process and should be removed prior to application (by spraying, if required).

Fit a mixing tip type 16 and extract material by pressing the lever. The material is completely and automatically mixed in the correct ratio in the mixing tip.

An intraoral tip type 2 may be inserted in the open end of the mixing tip type 16 to facilitate application.

Place the cap back on the cartridge after use. Before using it again, check the opening of the cartridge each time, and if it is blocked, remove all of the material blocking it and then extract a small amount of material, until both pastes are delivered uniformly.

2. Handling:

In case of occlusal bite registration, apply **Registrado X-tra** directly onto the upper or lower dental arch of the patient.

The working time of **Registrado X-tra** is ca. 30 s (from commencement of mixing at a room temperature of 73 °F/ 23 °C). After 40 s in the mouth **Registrado X-tra** can be removed.

Information, precautionary measures:

Avoid contact of the uncured silicone material with clothes. Disinfection of the registrations with an aqueous disinfectant is recommended.

Storage:

Store at temperatures of 39 °F - 73 °F (4 °C - 23 °C). Do not use after expiry date.

Ordering information:

Registrado X-tra

Pack 2 x 50ml safety cartridges, mixing tips type 16, intraoral tips type 2	REF 2035
Clinic pack 8 x 50ml safety cartridges, mixing tips type 16, intra oral tips type 2	REF 2036
Pack 50 mixing tips type 16	REF 2196
VOCO dispenser type 2	REF 2158
Pack 50 intraoral tips type 2	REF 2140

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith. Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

Keep this material out of reach of children.

For dental use only.

An explanation of the symbols used in labeling can be found at www.voco.dental/symbols

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VOCO GmbH Phone +49 (4721) 719-0