VOCO Structur® Premium

USA | EN Instructions for use

Carefully read instructions prior to use

Structur® Premium is a self-curing, highly aesthetic paste-paste system for the production of temporary crowns, bridges, inlays and onlays.

Structur® Premium has excellent mechanical properties and is fluorescent. **Structur® Premium** is available in 6 shades A1, A2, A3, A3.5, B1 and BL (Bleach).

Indications:

Temporary crowns, bridges, inlays, onlays.

Contraindications:

Structur® Premium contains mod. Bis-GMA, amines, benzoyl peroxide, BHT. Do not apply in patients with known allergies to these ingredients.

Instructions:

Insert **Structur® Premium** cartridge into the dispenser type 2 or a compatible dispenser with a suitable piston. Due to technical reasons, slight differences in the filling levels of the two cartridge chambers may occur. In this case, remove cartridge cap and squeeze the material out until it is delivered evenly from both chambers. Then, insert a mixing tip type 6 and lock with a 90° clockwise turn. By pressing the lever, the material is extruded and automatically mixed in the correct ratio.

Do not remove the mixing tip from the cartridge after use. Replace the used mixing tip with a new one before the next application. Check the cartridge opening to ensure material flowability when changing the mixing tips.

Application:

Level out any undercuts in the preparation. Use only eugenol-free cements. In cases where no prefabricated crowns or similar pieces are used, prepare an impression. Prepare the impression by levelling undercuts and cutting drainage channels.

If the wall thickness of the temporary restoration is too thin, enlarge the impression, especially removing septums in the interdental embrasures. Then, clean and air dry the impression.

Apply **Structur® Premium** directly into the impression. Do not overfill.


The impression must be applied in the mouth no later than 30 sec. after commencement of mixing. Remove the temporary work from the mouth during the elastic phase, 1:00 min - 1:30 min, after mixing.

Do not allow **Structur® Premium** to fully set in the mouth!

The setting process can additionally be monitored by checking the excess material in the mouth.

The setting time of non-used material (in mixing tips) **cannot** be compared to the setting time of the material in the mouth.

At a room temperature of 73 °F (23 °C) and 50% humidity, working times are as follows:

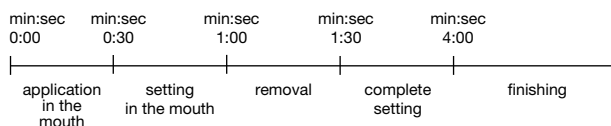Slight differences may result from different room temperatures.

After approx. 4 min, remove the temporary work from the impression and wipe dry. If necessary, use a solvent, e.g. ethyl alcohol. This removes the thin inhibition layer usually caused by oxygen and allows easier finishing. The surface of the temporary restoration should appear silky-matt.

Carefully trim and smooth any excess material and rough areas near the gingiva. Cement the temporary restoration using a eugenol-free temporary cement.

Caution:

- Before making the temporary piece, the prepared cores should be insulated for desensitization (fluoride varnishes, core varnishes, etc.)
- In order to avoid fractures or injuries from debris, the temporary should not be exposed to excessive loads (molar area).
- Linings and cores built up with resin-based materials (composites) should be sufficiently insulated (e.g. glycerine) for easy separation of the temporary from composite resin material.
- Inadequate hygiene or intensive use of some mouthrinses may lead to discoloration of the temporaries.
- **Structur® Premium** must be removed from the mouth during its elastic phase (not suitable for „shrink-to-fit techniques“)
- Finishing should only be started after complete setting of the material (approx. 5 min after mixing).
- The warranted product characteristics can only be achieved when using the supplied original mixing tips, which are also available as accessories.

Repairs:

If the temporary work breaks, despite **Structur® Premium's** extreme stability, the following procedures are recommended:

- **Air bubbles:** fill with **Structur® Premium**.
- In case of fracture immediately after preparing the temporary: re-connect the fracture with freshly mixed **Structur® Premium**.
- Restoration of previously used temporaries: Roughen the fracture with a bur and place undercuts. Apply a bonding agent to the prepared fractures and cure according to the respective instructions for use. Apply a flowable composite to the prepared areas. Bond the fractured pieces together and cure according to the instructions for use.

Storage:

Store at temperatures between 39 °F - 73 °F (4 °C - 23 °C).

Do not expose Structur® Premium to direct sunlight or the chairlight.

Product Ordering Information:

Structur® Premium	
pack, shade A1, cartridge 75gm, mixing tips type 6	REF 1712
pack, shade A2, cartridge 75gm, mixing tips type 6	REF 1713
pack, shade A3, cartridge 75gm, mixing tips type 6	REF 1714
pack, shade A3.5, cartridge 75gm, mixing tips type 6	REF 1717
pack, shade B1, cartridge 75gm, mixing tips type 6	REF 1718
pack, shade BL (Bleach), cartridge 75gm, mixing tips type 6	REF 1716
bulk pack, shade A3, 5 cartridges 75gm, mixing tips type 6	REF 1719
mixing tips type 6, 50 pieces	REF 2185
dispenser type 2	REF 2158

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use.

VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

Keep this material out of reach of children.

For dental use only.

An explanation of the symbols used in labeling can be found at www.voco.dental/symbols

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VOCO GmbH