Traypurol tabs

Carefully read instructions prior to use

Active-tabs for cleaning trays and instruments.

Traypurol tabs remove residue from alginates, zinc-oxide eugenol, carboxylate and phosphate cements from impression trays and instruments without using ultrasound. Traypurol tabs are bio-degradable.

Method of use:

4 Traypurol tabs are needed for ½ gallon (2 tabs for 1 litre) of cleaning solution.

Simply remove the tablet from the foil, place it in the cleaning container and fill with warm or cold water. Warm water accelerates the cleaning process. Avoid contact with skin.

Place trays and instruments in the tablet solution after removing any large pieces of material. Standing the trays upright promotes the cleaning process.

Leave impression trays and instruments in the solution for at least 2 hours, but not more than 72 hours. Remove and rinse with clean water.

Traypurol tabs are suitable for all rustproof metals and plastics.

Change the cleaning solution every 2 - 3 days.

Traypurol tabs can be used in ultrasonic baths, but not in dishwashers.

Information/Precautionary measures:

Contains complexing agent > 30%, non-ionic tenside < 5%, corrosion inhibitors. Irritates the eyes.

In case of contact with the eyes, rinse immediately with plenty of water and consult a physician. If swallowed, seek immediate medical attention and present package.

Store Traypurol tabs in a dry area at temperatures between 39°F - 73°F/ 4°C - 23°C.

Do not use in dishwashers.

PRODUCT ORDERING INFORMATION:

50 tabs 2355

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith. Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

Keep this material out of reach of children.

For dental use only.

An explanation of the symbols used in labeling can be found at www.voco.com/us/symbols

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VOCO GmbH Phone +49 (4721) 719-0