VOCO**USA**

VOCO Profluorid® Varnish

Carefully read instructions prior to use**Product description:**

VOCO Profluorid Varnish is a fluoride containing dental varnish for surface application to enamel and dentin. **VOCO Profluorid Varnish** will adhere to wet surfaces and is tolerant to moisture and saliva. **VOCO Profluorid Varnish** is an ethanolic suspension of colophony with artificial flavors and sweetened with xylitol.

1 ml of **VOCO Profluorid Varnish** contains 50 mg of sodium fluoride equivalent to 22.6 mg of fluoride.

Indications:

VOCO Profluorid Varnish is intended for use as

- Treatment of hypersensitive teeth
- Sealing the dentinal tubules for cavity preparations or on sensitive root surfaces
- Cavity liner under amalgam restorations

Contraindications:

VOCO Profluorid Varnish is contraindicated in patients with ulcerative gingivitis or stomatitis or known allergies to colophony.

VOCO Profluorid Varnish is not intended for systemic treatment.

Methods of Use:**Recommended dosage:**

- **Treatment of patients with primary dentition**
Use approx. 0.25 ml of **VOCO Profluorid Varnish**
- **Treatment of patients with adolescent or permanent dentition**
Use approx. 0.40 ml of **VOCO Profluorid Varnish**

VOCO Profluorid Varnish should be applied on a complete set of teeth in a very thin film thickness. A thick film thickness is more likely to peel off and does not provide better therapeutic results. A complete set of teeth can be treated between the ranges of 0.25ml and 0.40ml.

Application:

1. Thoroughly clean tartar from treatment areas.
2. For best results, eliminate any excessive moisture/saliva from area to be treated.
3. Use brush applicator or foam pellet to uniformly apply **VOCO Profluorid Varnish**, covering the entire surface to be treated as a thin film.
4. Allow area to become wet (either by gentle rinsing or natural salivary flow) to ensure setting of **VOCO Profluorid Varnish**.
5. Instruct patient to avoid hard food, alcohol, brushing or flossing for the next 4 hours after application.

Precautionary measures:

Edematous swellings have been reported on rare occasions especially following application to extensive surfaces. Direct ingestion should be avoided. Dyspnea associated with asthmatic children, has been reported for similar preparations in extremely rare cases. Patients having sensitive digestive systems may experience nausea, usually after extensive treatment. In case of adverse reactions, remove varnish with a soft toothbrush and floss, and then rinse with warm water.

No treatment with other fluoride preparations on day of application. Suspend treatment with fluoride tablets for several days after treatment. Do not swallow.

According to its purpose, **VOCO Profluorid Varnish** has a sticky consistency. Material remaining on the screw thread may cause the cap to stick. Wipe off any contaminating residues with an ethanol-containing cloth.

Storage:

Store at temperatures between 39°F - 73°F (4°C - 23°C). Do not use after expiration date.

PRODUCT ORDERING INFORMATION:

tube 10ml (melon)	REF 1267
tube 10ml (caramel)	REF 2232
tube 10ml (cherry)	REF 2233
tube 10ml (mint)	REF 2234
tube 10ml (cola lime)	REF 2236
50 SingleDose 0.25ml (melon), accessories	REF 1268
48 SingleDose 0.40ml (mixed), accessories	REF 2225
50 SingleDose 0.40ml (melon), accessories	REF 1269
50 SingleDose 0.40ml (caramel), accessories	REF 2226
50 SingleDose 0.40ml (cherry), accessories	REF 2227
50 SingleDose 0.40ml (mint), accessories	REF 2228
50 SingleDose 0.40ml (bubble gum), accessories	REF 2238
50 SingleDose 0.40ml (cola lime), accessories	REF 2240
200 SingleDose 0.40ml (melon), accessories	REF 1271
200 SingleDose 0.40ml (caramel), accessories	REF 2229
200 SingleDose 0.40ml (cherry), accessories	REF 2230
200 SingleDose 0.40ml (mint), accessories	REF 2231
200 SingleDose 0.40ml (bubble gum), accessories	REF 2239
200 SingleDose 0.40ml (cola lime), accessories	REF 2241

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

Keep this material out of reach of children.
For dental use only.

An explanation of the symbols used in labeling can be found at www.voco.dental/symbols
Manufactured by:

VOCO GmbH Phone +49 (4721) 719-0
Anton-Flettner-Str. 1-3 Fax +49 (4721) 719-140
27472 Cuxhaven e-mail: marketing@voco.com
Germany www.voco.dental

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