Carefully read instructions prior to use**Product description:**

VOCO Retraction Paste is a paste-like formulation in caps for direct application for temporary widening of the gingiva to provide a dry gingival sulcus. The astringent paste containing aluminum chloride can be applied directly from the cap. The paste can be used as an alternative or in combination with retraction cords and other retraction methods. Commercially available dispensers can be used for the application of **VOCO Retraction Paste**, e.g. the Caps Dispenser. One cap **VOCO Retraction Paste** is sufficient for up to three sulci and may only be used on one patient at a time.

Indications:

For temporary retraction of the marginal gingiva to provide a dry gingival sulcus when the periodontium is healthy prior to, e.g.

- analogue or digital impressions
- cementation of temporary and permanent restorations
- preparation of class II and V fillings

Contraindications:

VOCO Retraction Paste must not be used in patients with diseased periodontium, open furcations or exposed bone. **VOCO Retraction Paste** contains aluminum chloride and kaolin. **VOCO Retraction Paste** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients. Do not use in patients with active periodontitis.

Patient target group:

VOCO Retraction Paste is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

VOCO Retraction Paste should only be used by a professionally trained dental practitioner.

Use:

1. Bring material to room temperature before use. Remove the cap from the aluminum bag and place it in a suitable dispenser immediately before use.
2. Bend the tip of the cap into the required position, if necessary.
3. Dispense a small amount of paste onto a mixing pad and discard.
4. Rinse the sulcus thoroughly with water and dry it carefully from the side with an air stream (do not point the air stream directly into the sulcus).
5. Slowly apply the paste in a circular motion into the sulcus until the sulcus is filled and the paste excess comes out. To do this, the tip of the cap can be inserted into the sulcus up to the marginal epithelium, without pressure. To increase gingival retraction, an optional retraction cord can be inserted into the sulcus before applying the paste. When using retraction cuffs or caps, first apply the paste into the sulcus, insert the appropriate cuff or cap and then allow the patient to bite for 1 to 2 minutes.
6. The retention time of the **VOCO Retraction Paste** in the sulcus is 1 to 2 minutes. During this time, keep the treated area dry. **VOCO Retraction Paste** does not harden.
7. Afterwards, not later than 2 minutes, the paste must be completely removed from the mouth with an air-water spray. Use a suction device.
8. Carefully dry the prepared area from the side with the air stream, but do not point the air stream directly into the sulcus. If necessary, repeat the treatment from step 5 on.
9. Continue with the impression or restoration according to the manufacturer's instructions.

Warnings, precautionary measures:

- Wear safety goggles and protective gloves.
- Do not swallow **VOCO Retraction Paste**.
- In case of accidental eye contact, rinse immediately with plenty of water and consult a doctor if necessary.
- Use **VOCO Retraction Paste** caps for one patient only.
- Apply paste with even, not too strong pressure.
- Exposure time longer than 2 minutes may cause irritation of the gingiva.
- The oral mucosa may become whitish in contact with **VOCO Retraction Paste** due to protein coagulation. This is a reversible phenomenon, which disappears within a few days.
- Metal surfaces and instruments may corrode upon contact with **VOCO Retraction Paste**. Always rinse paste residues immediately with water.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Storage instructions and application method:

Storage at 39 °F – 73 °F (4°C-23°C). Bring material to room temperature before use. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **VOCO Retraction Paste** must be reported to VOCO GmbH and the responsible authority.

PRODUCT ORDERING INFORMATION

Caps 25 × 0.3 g
Caps 100 × 0.3 g

REF 1012
REF 1013

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use.

VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

Keep this material out of reach of children.

For dental use only.

An explanation of the symbols used in labeling can be found at www.voco.dental/symbols

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VOCO GmbH Phone +49 (4721) 719-0