Vococid

Etching liquid/gel

Carefully read instructions prior to use

Indications:

Vococid is intended to etch only that part of the enamel and of the dentin resp. (in total-etch-technique), which will be covered by a composite material:

- for composite fillings
- for adhesive fixing of Maryland bridges, inlays, brackets and temporary splinting
- for sealing for caries prophylaxis
- for linings to create a fixed bond (according to manufacturer's instructions)

Vococid is both simple to use and easily visible on the enamel due to the blue coloring and consistency of the preparation. With **Vococid Gel** an exact application is possible (e. g. for etching the enamel margins), the liquid is suitable for treatment of larger surfaces. It is best applied using foam pellets or a brush.

Contraindications:

Caution! Vococid contains phosphoric acid (liquid 36.1 %, gel 35 %!)

Precautions:

Wear protective gloves and safety glasses!

In case of contact with the eyes, rinse thoroughly with water.

Avoid contact with gingiva, certain composite filling materials or neighbouring teeth. If total etching is not used, cover the dentin before etching, if necessary. It is recommended to cover adjacent teeth and etched enamel surfaces, which are not covered with composite material, once or twice with a fluorid varnish for remineralization.

Without total etching (only etching of enamel):

Coat the surface of the enamel, which must be completely moist, with one or two applications of etching liquid/gel on a foam pellet. If necessary, apply extra etching liquid/gel.

Vococid should be allowed to act for the recommended periods of time as shown below:

approx. 20 sec. for normal enamel and for enamel of adolescents, approx. 30 sec. for devitalized teeth, deeply fluoridated enamel or enamel with lack of prisms (e.g. primary teeth).

After application rinse thoroughly with water, spraying each etched tooth for at least 15 - 30 sec. Afterwards the patient should not close his/her mouth. Dry the etched surfaces thoroughly using an air blower.

Absolutely avoid contamination with saliva. In case of contamination, repeat the etching process (10 sec.). Control the etched areas (chalky-white).

Total-etch-technique with dentin and enamel bonding materials (etching of enamel and dentin):

Etch entire cavity (enamel appr. 20 - 30 sec., dentin max. 15 sec.). Do not apply more than 15 sec. on dentin. Suck away **Vococid** etching gel/liquid, rinse and remove excess moisture with an air-blower.

With other preparations, please refer to the manufacturer's instructions for use.

Storage:

Store **Vococid** at room temperature (39° - 73°F = 4° - 23°C) and close thoroughly after use. Do not use after the expiration date.

Product ordering information:

Gel syringe 5ml, application cannulae type 42	REF 1063
Gel syringe 5 x 2ml, application cannulae type 48	REF 1224
Liquid bottle 2 x 3ml	REF 1066

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

Keep this material out of reach of children.

For dental use only.

An explanation of the symbols used in labeling can be found at www.voco.dental/symbols

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VOCO GmbH Phone +49 (4721) 719-0